 **PaX-i3D** *Smart*  
(PHT-30LFO)

User Manual | Version 1.2.0

ENGLISH

FRANÇAIS

DEUTSCH

ITALIANO

ESPAÑOL

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## Notice

Thank you for purchasing the **PaX-i3D Smart** extra-oral imaging system.

The user manual is a part of product.

This manual describes how to operate the **PaX-i3D Smart** system. It is recommended that you thoroughly familiarize yourself with this manual in order to make the most effective use of this equipment. Observe all cautions, safety messages and warnings which appear in this manual.

The information contained in this manual is subject to change without prior notice to the persons concerned. For the most current information, please contact us at:

**VATECH Co., Ltd.**

**Phone: +82-1588-9510**

**E-mail: [gcs@vatech.co.kr](mailto:gcs@vatech.co.kr)**

**Website: [www.vatech.co.kr](http://www.vatech.co.kr)**

**Manual Name: PaX-i3D Smart (Model: PHT-30LFO) User Manual**

**Version: 1.2.0**

**Publication Date: 2015-2**

**Copyright © 2015 by VATECH**

All rights reserved.

The documentation, brand name and logo used in this manual are copyrighted.

No part of this manual may be reproduced, transmitted, or transcribed without the expressed written permission of the manufacturer.

We reserve the right to make any alterations which may be required due to technical improvement.

## Conventions in this Manual

The following symbols are used throughout this manual. Make sure that you fully understand each symbol and follow the instructions which accompany it.

To prevent personal injury and/or damage to the equipment, please observe all warnings and safety information included in this document.

	<b>WARNING</b>	Indicates information that should be followed with the utmost care. Failure to comply with a warning may result in severe damage to the equipment or physical injury to the operator and/or patient.
	<b>CAUTION</b>	Indicates a situation that demands prompt and careful action, a specific remedy, or emergency attention.
	<b>X-RAY</b>	Indicates a possible danger of exposure to radiation.
	<b>IMPORTANT</b>	Indicates a situation or action that could potentially cause problems to the equipment and/or its operation.
	<b>NOTE</b>	Emphasizes important information or provide useful tips and hints.
	<b>SINGLE USE</b>	Indicates a component which must be replaced for each new patient.

# 1 PaX-i3D Smart Imaging System Overview

The **PaX-i3D Smart** is an advanced digital dental diagnostic system that incorporates Panoramic (PANO), Cephalometric (CEPH), and 3D CT imaging capabilities into a single system.

The **PaX-i3D Smart** system is intended for use to take a panoramic, cephalometric and CT image of the oral and craniofacial anatomy to provide diagnostic information for adult and pediatric patients.

The **PaX-i3D Smart** may only be used by dentists, X-Ray technicians and other professionals who are licensed to perform X-Rays by the law of the region in which it is being used.

## Standards and Regulations

The PaX-i3D Smart is designed and manufactured to meet the following standards:

IEC/EN 60601-1(3rd), UL 60601-1(1st), IEC/EN 60601-1-3, IEC 60601-2-63

21 CFR 1020.30, 31, 33

NEMA Standard publication PS 3.1-3.18, 2008

The CE symbol grants this equipment compliance with the European Directive for Medical Devices 93/42/EEC as amended by 2007/47/EC as a class IIb device.

UL symbol grants this equipment compliance with the UL 60601-1 / CAN/CSA C22.2 No.601.1


## Classifications (IEC60601-1 6.1):

Protection against the ingress of water: Ordinary Equipment: IPX0

Protection against electric shock: Class 1 equipment, Type B Applied Parts

## 1.1 General View of the PaX-i3D Smart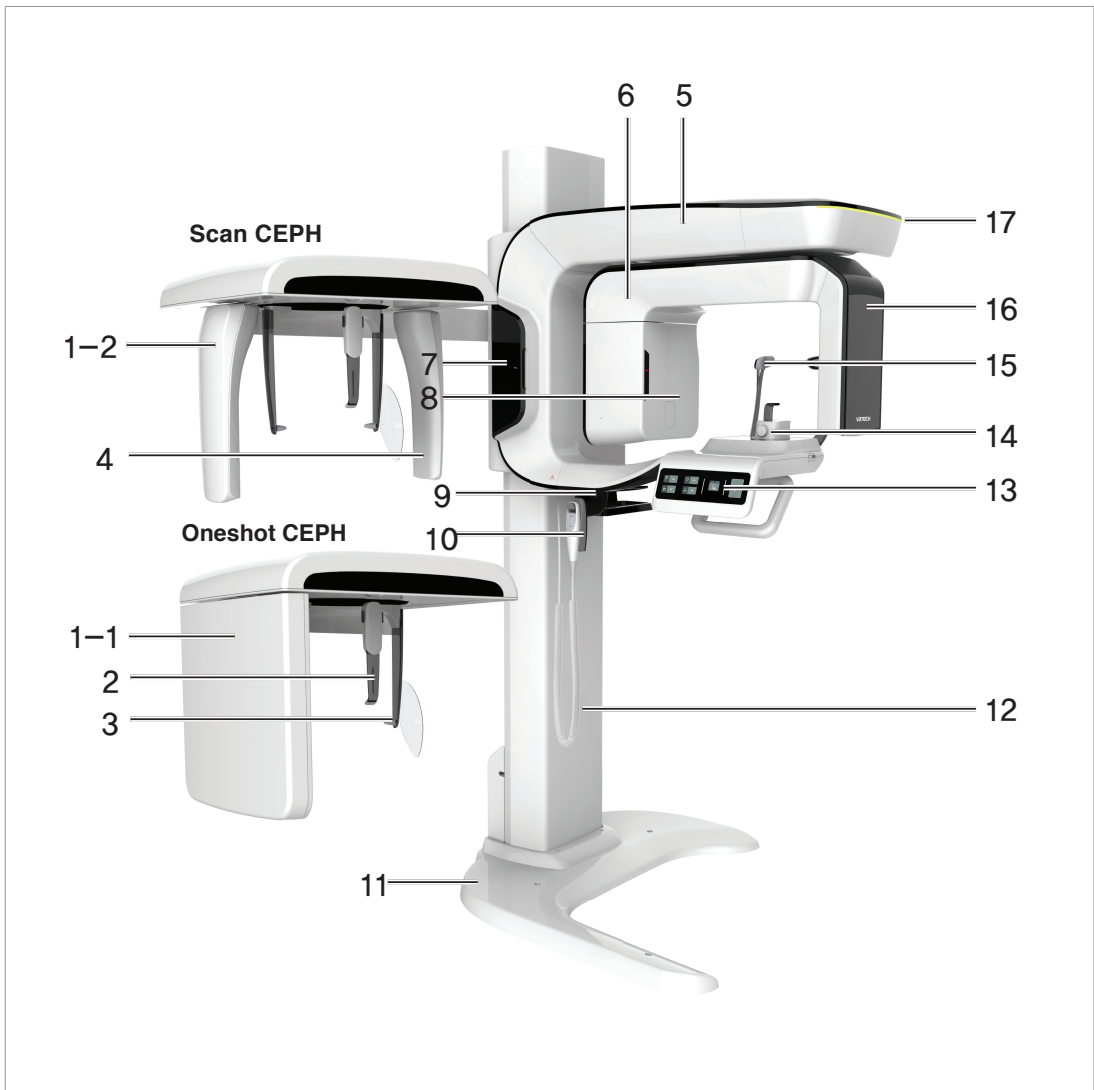

No.	Item	Details
1	<b>X-RAY SENSOR (CEPH)</b>	CEPH imaging sensor (optional). - 1-1. One shot Type - 1-2. Scan Type
2	<b>NASAL POSITIONER</b>	- Positions the patient during CEPH imaging. - The ruler for use to compensate the acquired image that is different from actual size.
3	<b>EAR ROD</b>	Secures the patient's head during CEPH imaging.
4	<b>SECONDARY COLLIMATOR</b>	Limits the X-Ray irradiation field for CEPH scanning.
5	<b>Vertical Frame</b>	Holds the Rotating Unit. Use the Vertical Frame Up/Down switch.
6	<b>ROTATING UNIT</b>	Rotates around the patient's head during exposure (depending on the imaging mode).
7	<b>Enclosed Component Storage</b>	Place where bite blocks, integrated chin rests and the other components are stored.
8	<b>X-RAY TUBE</b>	The source of X-Ray emission.
9	<b>Emergency Switch</b>	Powers off the equipment when there is a problem during operation.
10	<b>Vertical Frame Up/Down Switch (optional)</b>	Adjusts the height of the vertical frame.
11	<b>BASE (optional)</b>	Used to balance and stabilize the equipment.
12	<b>STATIONARY COLUMN</b>	The portion of the column fixed to the base unit.
13	<b>CONTROL PANEL</b>	Allows the operator to control certain functions of the equipment and displays operational parameters.
14	<b>CHINREST</b>	Supports the patient's chin during image acquisition.
15	<b>TEMPLE SUPPORT</b>	Supports the patient's temples during PANO scan.


No.	Item	Details
16	<b>X-RAY SENSOR (PANO / CT)</b>	Sensors: used for CT and PANO imaging.
17	<b>LED Lamp</b>	Indicates the emission status while the x-ray is in operation. - Green: Ready - Orange: X-ray ON

NOTE

**The structural overview of the PaX-i3D Smart may differ depending on the model.**

## 1.2 Control Panel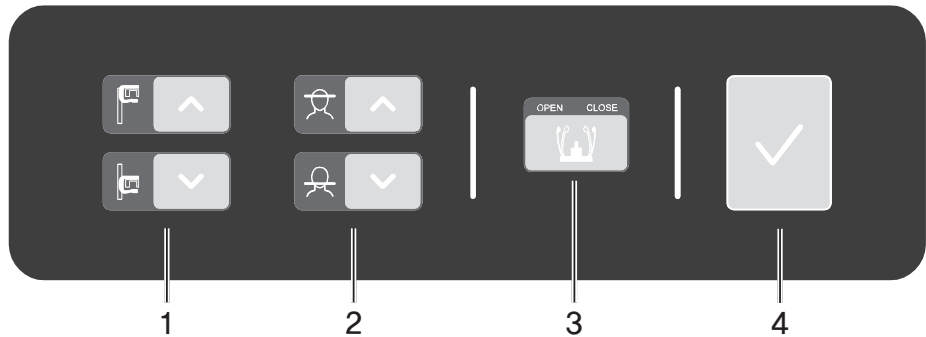

Buttons		Functions
<b>1</b>	Vertical Frame Up/Down button	Adjusts the vertical frame by moving vertically.
<b>2</b>	Frankfurt plane laser beam control button	Positions the frankfurt plane laser beam in pano mode.
<b>3</b>	Temple Support OPEN/CLOSE button	Adjusts the temple supports to position the patient.
<b>4</b>	READY/ RETURN button	When pressed, prepares for operation after positioning the patient and configuring the environment settings.  Initializes the positioning of the rotating unit.

## 2 Getting Started

### 2.1 Turning on the PaX-i3D Smart



**Do not position the patient in the unit while it is initiating. The patient could be injured if the equipment malfunctions.**

- Condensation could form inside the equipment if it is a different temperature than the surrounding room. Only turn on the equipment once it has reached room temperature.
- Wait at least 20 seconds after the equipment has been turned off to turn it back on.
- Allow the equipment to warm up for at least 5 minutes before acquiring images or preferably more than 30 minutes for image quality.

1. Turn the **ON / OFF** switch underneath the handle frame to the ON position.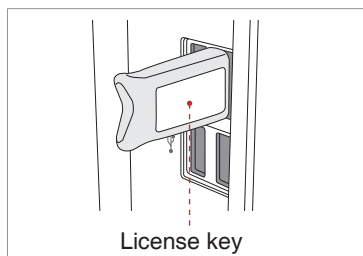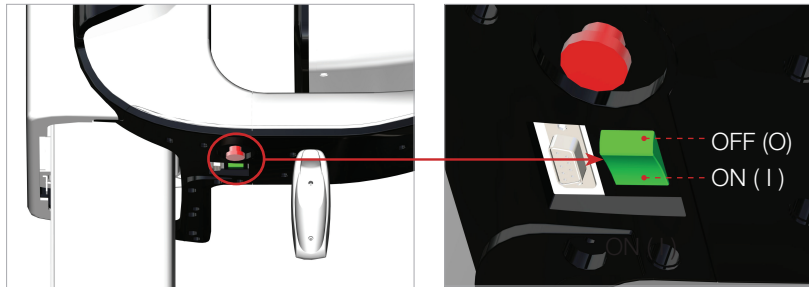

2. Insert the license key for **Ez3D plus** into the USB port on the back of PC. The license key for **Ez3D plus** must be attached to view or analyze the 3D images.

## 2.2 Running the Image Viewer

**EasyDent / EzDent-i** is a basic imaging platform for all VATECH's dental X-Ray equipments. The Imaging Program is interfaced with **EasyDent / EzDent-i**. On your desktop, double-click **EasyDent / EzDent-i** icon. The **EasyDent / EzDent-i** main window will be displayed.

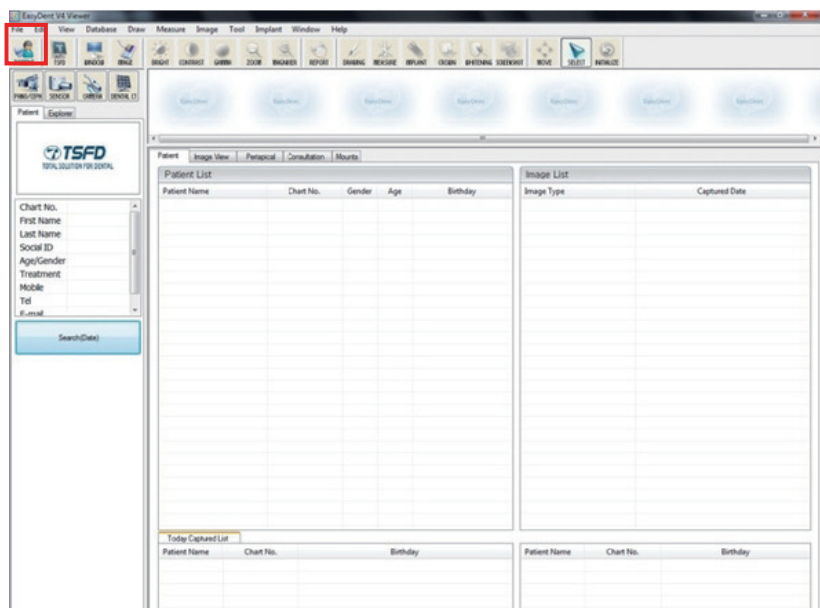
For further details on this subject, refer to the **EasyDent / EzDent-i user manual**.

### 2.2.1 Creating a New Patient Record

To create a new patient record, follow the procedure outline below:

#### EasyDent

1. Click the **Patient** icon () on the upper left corner of the EasyDent's main GUI window.

2. The following dialog box will open.

3. Enter the required patient information. **Chart Number, First Name, and Last Name** are required fields which must be filled in. All other fields are optional, but it is recommended that they be filled in.

4. Click **Add** to save the patient record.

### EzDent-i

1. Click the **Add Patient** icon from main GUI window.

2. Enter the required patient information. The **Chart Number, E-Mail address, First Name, and Last Name** are required fields which must be filled in. (The chart number fills in automatically.)


3. Click **Add** to save the patient record.

## 2.2.2 Retrieving Patient Records

### EasyDent


You can search through the patient database using a patient's chart number, their first name, or their last name.

1. On the **Patient information pane**, double-click the **Chart No.**, **First name**, or **Last name** of the patient and the virtual keyboard will pop up.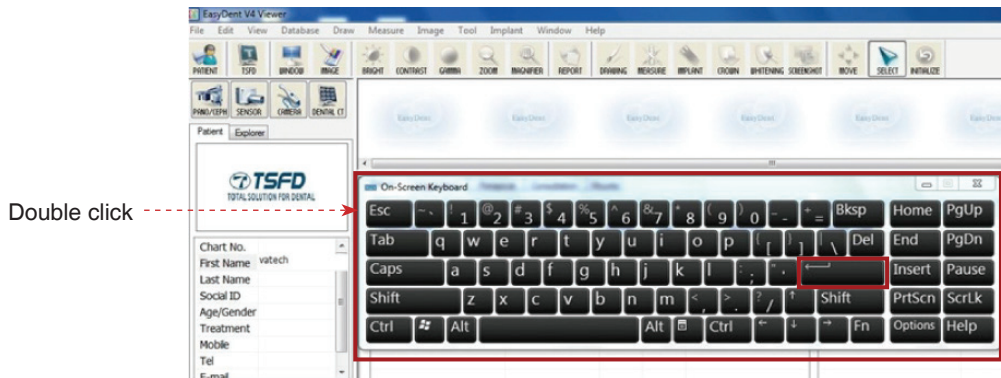


2. Enter the **Chart No.**, **First name**, or **Last name** of the patient by clicking the mouse on the virtual key board and click the **Enter**.

3. Patient information can be displayed on the **Patient information pane** and **Patient List**.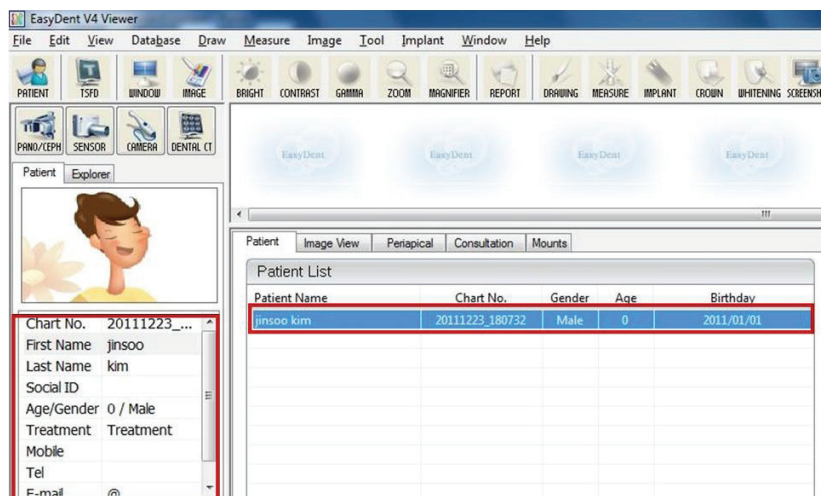

## EzDent-i


1. Enter the name or chart number of the patient to be searched on the **Search** pane and then click the **Search** button. The information on the patient that fits the search condition appears.

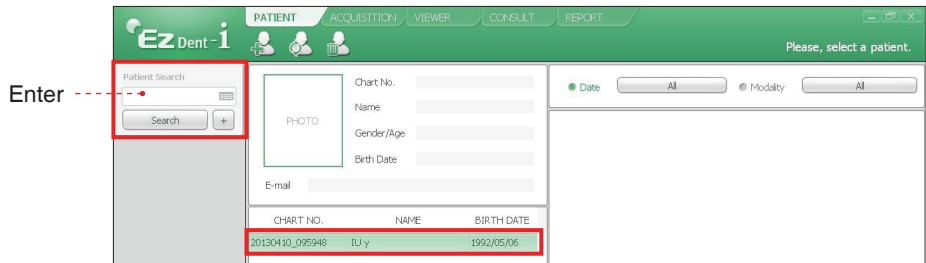Double-click the Keyboard icon to display the virtual keyboard. You may search patient information using the virtual keyboard.

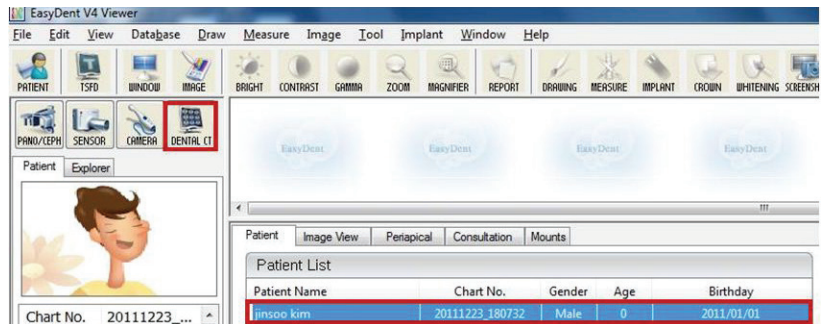
2. Double-click the patient information to see more details about the patient as shown below.

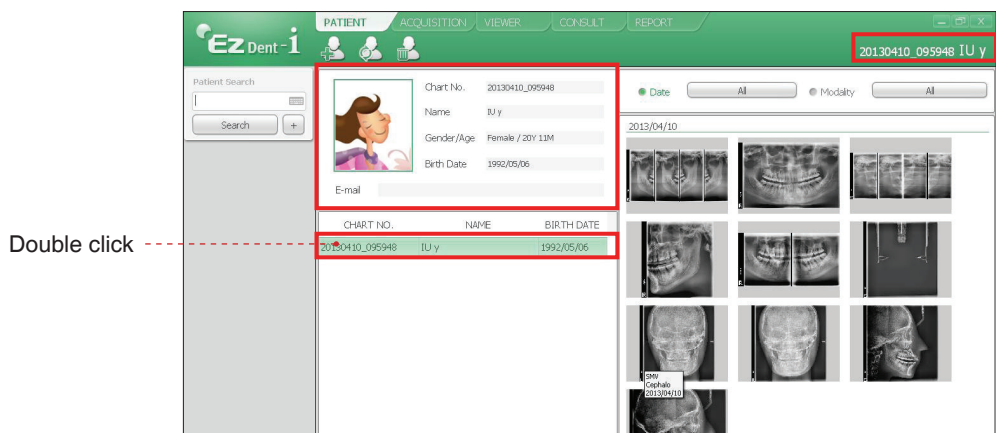## 2.3 Initiating the Imaging Program

### EasyDent

1. First, click the patient information in the patient list, and click the **Dental CT** icon (  ) in the upper left corner of the EasyDent's main window to open the imaging program.


2. The following imaging program window opens. The sole purpose of this window is to control equipment settings and acquire images.


## EzDent-i

1. Search and select the patient to be captured.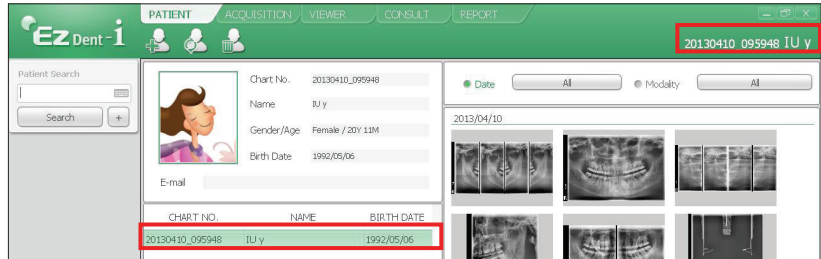

2. Click **Acquisition** and the imaging mode (**CT, Panorama, or Cephalo**).

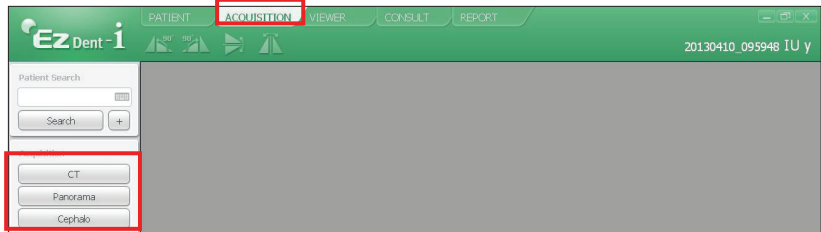3. The main screen for the selected mode appears. From the main screen, you can configure the imaging parameter settings prior to acquiring an image.

Please proceed to the next chapter.


NOTE

Refer to chapters 3 - 5 for information regarding image acquisition.

## 3 Acquiring PANO Images

### 3.1 Setting Exposure Parameters

Perform the following procedures to select the capture parameters for the specific patient and capture mode.

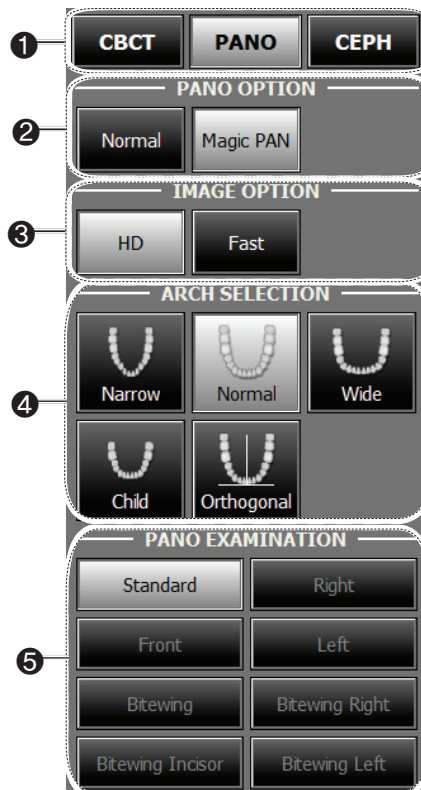1. Click the PANO button.

2. Select a PANO option.

Mode	Details
<b>Normal</b>	Normal image.
<b>Magic PAN (Optional)</b>	Reconstruct an image with optimized focus (autofocus) from the entire panorama to acquire high quality images so any errors caused by issues with the patient's position and tooth trajectory will be minimized.

3. Select an Image option.

Mode	Details
<b>HD</b>	High Definition, High Resolution
<b>Fast</b>	Normal Image


**You can select an image option only when you have purchased Magic PAN option. Default image option is HD.**

4. Select the patient's arch type. By default, **Standard** is selected.

**Orthogonal:** This mode enables overlapping regions of teeth to be minimized when acquiring images in the ROI. If **orthogonal** arch is selected, its sub-modes are activated.

5. Select the ROI for panoramic image acquisition under **PANO Examination**.

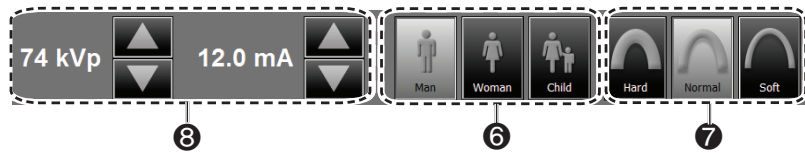6. The patient's gender will be automatically selected according to the patient information registered in EasyDent / EzDent-i. Nevertheless, ensure that this information is correct.

Age Group		VATECH's Standard
	Child	$\leq 12$
Adult	Man	$\geq 13$
	Woman	

7. Select X-ray intensity.

**X-ray intensity (Hard, Normal, Soft) is according to the operator's decision.**  
**Soft  $\leq$  Normal  $\leq$  Hard**

8. A default value for the tube voltage (kVp) and current (mA) will be displayed based on the gender and X-ray intensity. If necessary, further adjustments can be made using the arrows to the right of each number.

9. Click the **CONFIRM** button for these parameters to take effect.

NOTE

Please wait for a moment, while the rotating unit moves to its initial scanning position.

When you click the **CONFIRM** button,

- The **READY** button will begin blinking to show that it has been activated.

- The rotating unit will move to its initial scanning position.
- Three laser beams, **Mid-sagittal plane, Frankfurt plane and Canine**, will be activated to facilitate patient positioning.
- The scan time and DAP (Dose Area Product) values will be shown on the main display for the selected exposure.

Scan Time  
13.6 Sec



DAP  
128.775880 mGy x Cm<sup>2</sup>

10. Guide the patient to the equipment.

## 3.2 Positioning the Patient

### Before Positioning the Patient

- Ask the patient to remove all jewelry and metallic objects, such as earrings, hair pins, spectacles, dentures, and orthodontic appliances. These items may cause shadow images which could obscure diagnosis.
- It is strongly recommended that the patient wear a lead apron for protection against any possible scatter radiation.
- Adjust the equipment to the patient's height by using the Vertical Frame up/down button or switch.

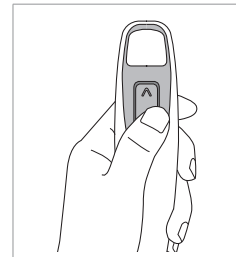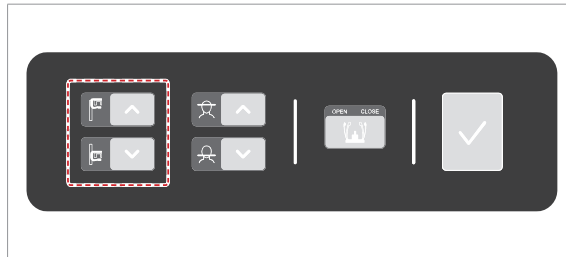**Vertical Frame  
Up/Down  
switch(optional)**

NOTE

**Correct positioning is an important factor in capturing the best possible image. Proper positioning will reduce the appearance of the cervical spine in the image.**

CAUTION

**Disinfect the chin support and bite block using an alcohol-based cleaning solution and wipe away all residue with dry cloth.**

**If you don't adjust the laser beams for patient alignment (vertical, horizontal, and canine beams), part of the image may be zoomed in or out or a ghost may appear in the image.**


WARNING

**Be careful not to project the laser beams directly into the patient's eyes as this could severely damage the patient's vision.**

### 3.2.1 PANO Standard and Bitewing mode

#### Positioning the patient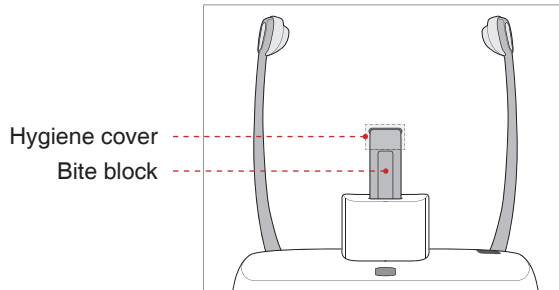


1. Insert the bite block into the Chinrest receptacle on the equipment.
2. Place a hygienic cover over the bite block.

**The hygiene cover for the bite block and chin support is intended for single use only. Always replace the hygiene cover for each new patient.**

**Disinfect the Chinrest, Chin support and bite block using an alcohol-based cleaning solution and wipe away all residue with dry cloth.**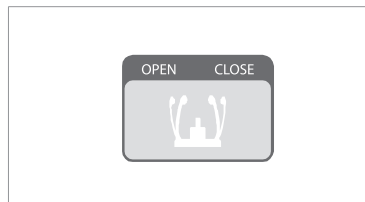


3. Open the temple supports by clicking the temple support adjustment button.

4. Guide the patient —facing the Chinrest — to the equipment.
5. Adjust the equipment to the patient's height by using the Vertical Frame up/down button or switch until the patient's chin rests comfortably on the Chinrest.

6. Ask the patient to:

- Stand upright
- Firmly grip the handle
- Lean his/her chest lightly against the equipment
- Position his/her feet slightly forward and open stance

7. Ensure that the patient's shoulders remain level and that his/her neck is relaxed. The cervical spine should be straight and upright.

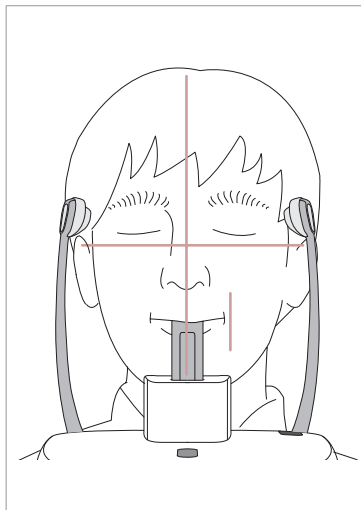8. Have the patient bite the bite block along its grooves using his/her upper and lower incisors

9. Have the patient

- Close his/her lips around the bite block
- Keep his/her tongue pressed against his/her palate
- Close his/her eyes

Ask the patient to remain still until scanning is completed.


CAUTION

To acquire the best image possible, ask the patient not to:


- Breathe or swallow saliva during image acquisition
- Move during image acquisition


### **Positioning the edentulous patient**

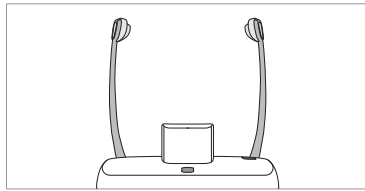1. Remove the bite block from the Chinrest receptacle on the equipment.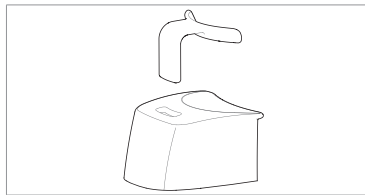

2. Insert the Sinus/Edentulous Bite into the chinrest.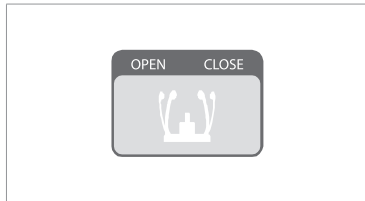


3. Open the temple supports by clicking the template support adjustment button.

4. Guide the patient —facing the Chinrest — to the equipment.

5. Adjust the equipment to the patient's height by using the Vertical Frame up/down button or switch until the patient's chin rests comfortably on the Chinrest.

6. Ask the patient to:

- Stand upright
- Firmly grip the handle
- Lean his/her chest lightly against the equipment
- Position his/her feet slightly forward and open stance

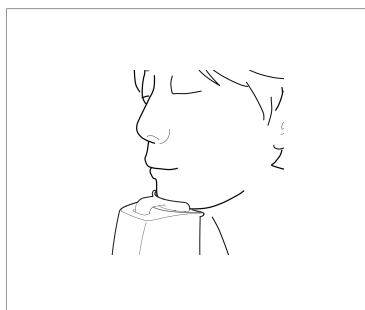7. Have the patient

- Close his/her lips around the bite block
- Keep his/her tongue pressed against his/her palate
- Close his/her eyes


Ask the patient to remain still until scanning is completed.

## Aligning the Laser Beams


**Be careful not to project the laser beams directly into the patient's eyes as this could severely damage the patient's vision.**


- 1. Mid-sagittal plane laser beam:** Position the Mid-sagittal plane laser beam on the center of the patient's face to prevent magnifications of the left or right side in the final image.
- 2. Frankfurt plane laser beam:** Position the patient's head so that the Frankfurt plane is aligned with the Frankfurt plane laser beam.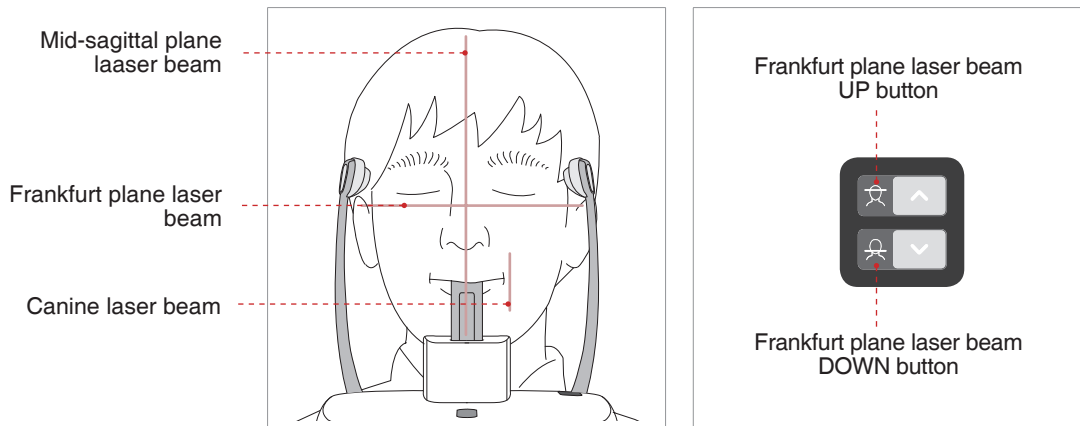

- 3. Canine laser beam:** Have the patient smile to properly position the canine laser beam on the center of the patient's canine tooth.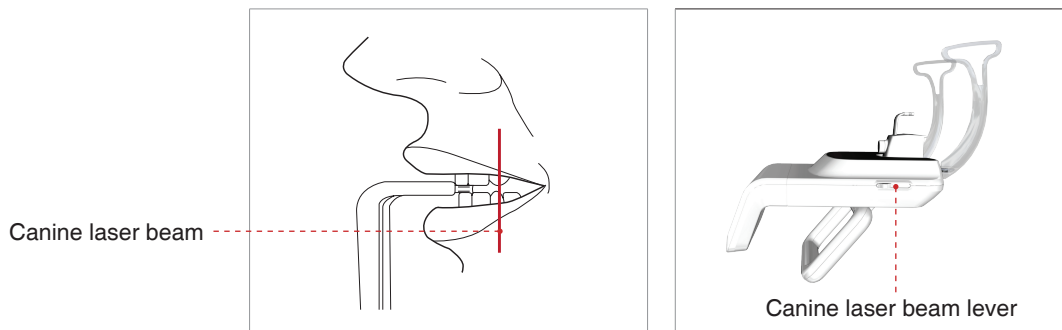

### Completing Patient Positioning

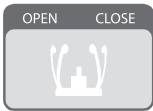1. After confirming patient positioning and beam alignment, adjust the temple supports to fit snugly on both sides of the patient's head by clicking the temple support adjustment button.

**Make sure that the Temple Support is in Close position before clicking the READY button.**

2. Click the **Ready** button after the patient has been properly positioned. No X-Ray will be emitted at this point.
3. Proceed to section **3.3 Initiating X-Ray Exposure**.

### 3.2.2 TMJ Open


There are two TMJ sub modes: **TMJ Open** and **TMJ Close**. Scan a TMJ Open image first and then scan a TMJ Close image. To prepare and position the patient, follow the process outlined below.

**For a child with a large head circumference, Man/Woman mode may be selected instead of Child.**

### Positioning the patient

1. Remove the standard chinrest and insert the sinus chinrest into the equipment.

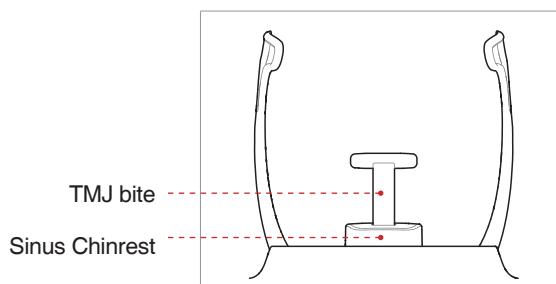2. Insert the TMJ bite into the sinus chinrest.

**Disinfect the Chinrest using an alcohol-based cleaning solution and wipe away all residues with dry cloth before proceeding any further.**

3. Open the **Temple Supports** by clicking the temple support adjustment button.


4. Guide the patient —facing the Chinrest — to the equipment.

5. Adjust the equipment to the patient's height by using the Vertical Frame up/down button or switch until the patient's chin rests comfortably on the Chinrest. Ensure that the chin is in contact with the Chinrest.

6. Ask the patient to:

- Stand upright
- Firmly grip the handle
- Lean his/her chest lightly against the equipment
- Position his/her feet slightly forward.

7. Have the patient place his / her acanthion point on the chin support and tilt his / her head forward by 5 degree.

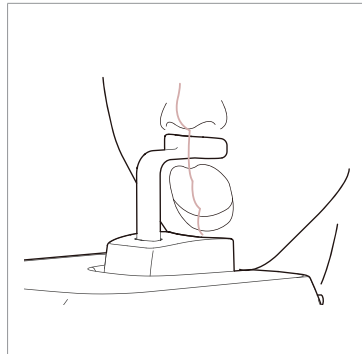8. For **TMJ Open** imaging, have the patient:

- Open his/her mouth as wide as possible
- Keep his/her tongue positioned towards the palate
- Close his/her eyes.
- Be careful the chin not to touch any part of the equipment.

9. Ensure that the patient's shoulders remain level and that his/her neck is relaxed. The cervical spine should be straight and upright. Ask the patient to remain still until scanning is complete.

To acquire the best image possible, ask the patient not to:


- Breathe or swallow saliva during image acquisition
- Move during image acquisition

## Aligning the Laser Beams

Be careful not to project the laser beams directly into the patient's eyes as this could severely damage the patient's vision.

1. **Mid-sagittal plane laser beam:** Position the Mid-sagittal plane laser beam at the center of the patient's face to prevent magnifications of the left or right side in the final image.
2. **Frankfurt plane laser beam:** Position the patient's head so that the Frankfurt plane is aligned with the Frankfurt plane laser beam.

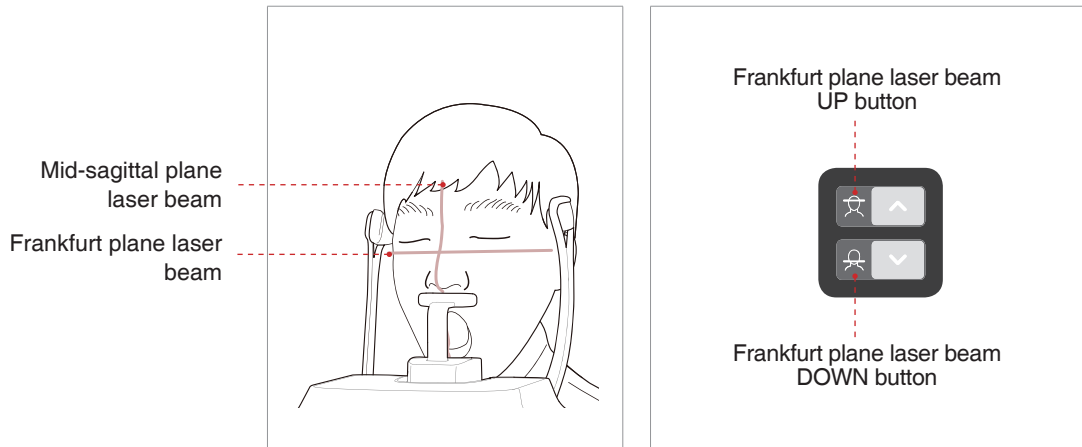## Completing Patient Positioning

1. After confirming patient positioning and beam alignment, adjust the temple supports to fit snugly on both sides of the patient's head by clicking the temple support adjustment button.

**Make sure that the Temple Support is in Close position before clicking the READY button.**


2. Click the **Ready** button after the patient has been properly positioned. No X-Ray will be emitted at this point.
3. Proceed to section **3.3 Initiating X-Ray Exposure**.
4. When the Temple Support is open, guide the patient out of the equipment.

### 3.2.3 TMJ Close

**Scan a TMJ Open image first and then scan a TMJ Close image. Guide the patient to close his/her lips for TMJ Close imaging while maintaining the same position used for TMJ Open imaging.**

1. Shortly after **TMJ Open** imaging has been completed, you will receive the following message: "Do you want to take the **TMJ Close** image continuously?" Press the **OK** button to capture a **TMJ Close** image.
2. Guide the patient to the inside of the equipment.
3. Have the patient place his / her acanthion point on the chin support and tilt his / her head forward by 5 degree.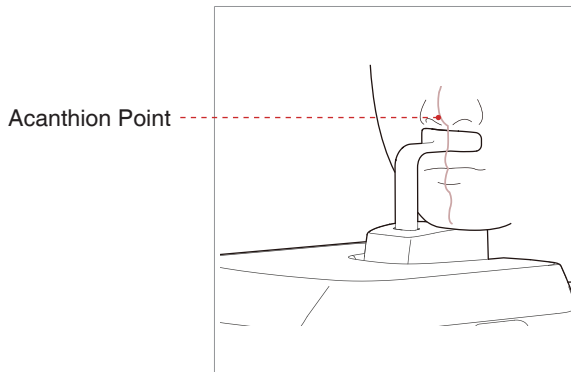

4. For **TMJ Close** imaging, Have the patient:


- Close his/her mouth
- Keep his/her tongue pressed against his/her palate
- Close his/her eyes

5. The laser beams should be aligned in the same way as for **TMJ Open** imaging.

6. Proceed to section **3.3 Initiating X-Ray Exposure**.

### 3.2.4 Sinus

#### Positioning the Patient

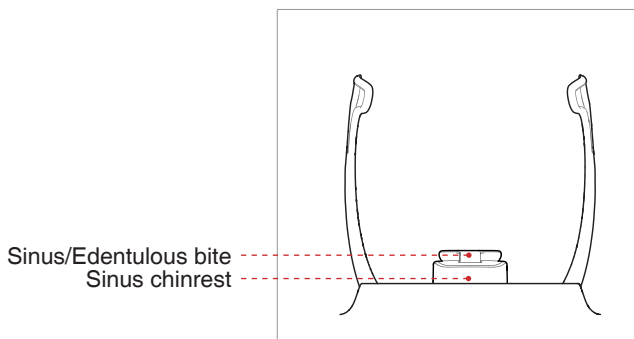
1. Insert the sinus chinrest into the equipment.

2. Insert the Sinus/Edentulous bite into the sinus chinrest.

**Disinfect the Chinrest using an alcohol-based cleaning solution and wipe away all residues with a dry cloth before proceeding any further.**

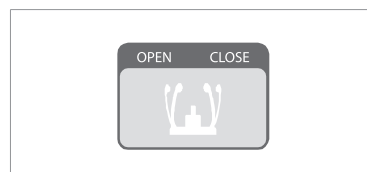3. Open the **temple supports** by clicking the **temple support adjustment button**.


4. Guide the patient —facing the Chinrest — to the equipment. Have the patient stand upright in the center of the equipment.

5. Adjust the equipment to the patient's height by using the Vertical Frame up/down button or switch until the patient's chin rests comfortably on the Chinrest.

6. Ask the patient to:

- Stand upright
- Firmly grip the handle
- Lean his/her chest lightly against the equipment
- Position his/her feet slightly forward.

7. Ensure that the patient's shoulders remain level and that his/her neck is relaxed. The cervical spine should be straight and upright.

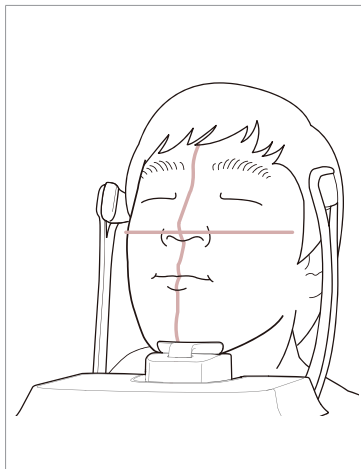8. Have the patient:

- Tilt his/her head back by 10-15°
- Close his/her mouth
- Keep his/her tongue pressed against his/her palate
- Close his/her eyes

Ask the patient to remain still until scanning is completed.

To acquire the best image possible, ask the patient not to:

- Breathe or swallow saliva during image acquisition
- Move during image acquisition


## Aligning the Laser Beams

**Be careful not to project the laser beams directly into the patient's eyes as this could severely damage the patient's vision.**

- 1. Mid-sagittal plane laser beam:** Position the Mid-sagittal plane laser beam at the center of the patient's face to prevent magnifications of the left or right side in the final image.
- 2. Frankfurt plane laser beam:** The Frankfurt plane laser beam should go from the top of the ear to the tip of the nose when the patient's head is tilted back 10 – 15 °.

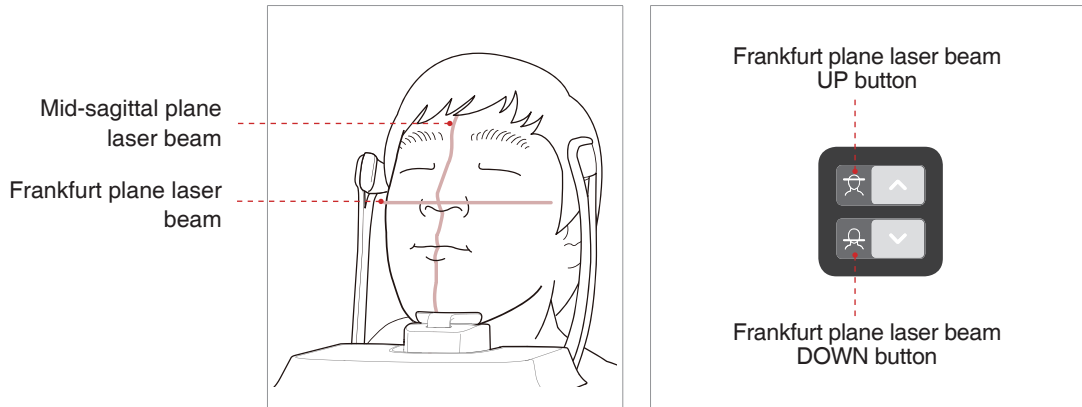### Completing Patient Positioning

1. After confirming patient positioning and beam alignment, adjust the temple supports to fit snugly on both sides of the patient's head by clicking the temple support adjustment button.

CAUTION

**Make sure that the Temple Support is in Close position before clicking the READY button.**

2. Click the **Ready** button after the patient has been properly positioned. No X-Ray will be emitted at this point.

3. Proceed to section **3.3 Initiating X-Ray Exposure**.

## 3.3 Initiating X-Ray Exposure

The X-Ray exposure method and its sequences are the same for PANO Standard and PANO Special modes. The example used in this manual is the PANO Standard mode.

To emit the X-Ray, perform the following procedure.

WARNING

- **Do not operate the PC during exposure. Failure to comply with this instruction may cause the system to malfunction.**
- **The operator shall observe the X-Ray safety regulations applicable to their area at all times during the operation of this equipment.**

CAUTION


**If an emergency occurs during image acquisition, release the exposure switch to cease X-Ray emission.**

IMPORTANT

**Direct the patient to not move during the exposure until the Temple support is open.**

1. Leave the X-Ray room and close the door. The operator must visually monitor the patient at all times during image acquisition.
2. Press and hold down the exposure switch until image acquisition is complete.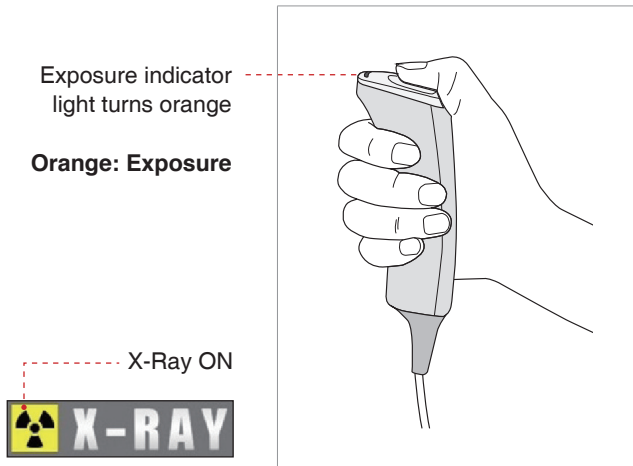

During X-Ray irradiation, ensure that

- The LED lamp on top of the equipment turns orange to indicate X-Ray emission.
- The warning lamp outside the X-Ray room turns on.
- The sound (beep or music: optional) goes off.
- The radiation symbol on the upper left corner of GUI turns yellow to indicate X-Rays are being emitted.

3. Release the exposure switch when “Image capturing is completed” message appears on the screen.
4. The image appears in real time on the imaging GUI.
5. Click the **Save** button to save the image.

### After Image Acquisition

After acquiring the image, complete the following steps:


- Loosen the temple supports to release the patient.
- Remove the hygiene cover from the bite block (for **standard PANO** mode only).
- Press **Return** to bring the Rotating Unit back to its initial position.

## 4 Acquiring CEPH Images

### 4.1 Setting the Exposure Parameters

Perform the following procedures to set the exposure parameters for the specific patient and capture mode.

Depending on the sensor type employed, one of three kinds of imaging S/W comes with the equipment for the CEPH mode examination.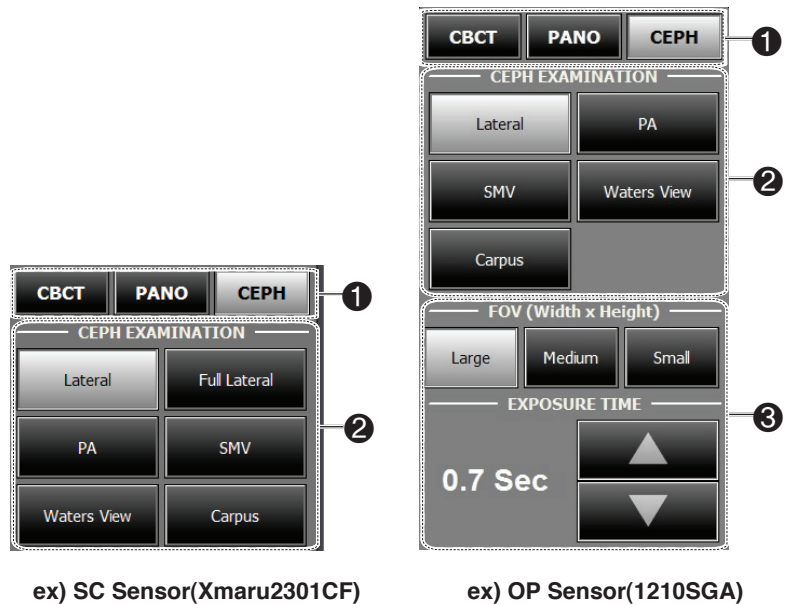


1. Click the **CEPH** button.
2. Select the scanning mode under **CEPH Examination**.

3. (For One-shot CEPH only) Do the selections that follow.

- **FOV Size:** Select the FOV size.

FOV	Details
<b>Large</b> : 12x10 (inches) : 30.48x25.40 (cm)	Full size
<b>Medium</b> : 9x10 (inches) : 22.86x25.40 (cm)	Region of the no interest of the rear part of the head is eliminated to minimize the X-ray exposure area.
<b>Small</b> : 8x8 (inches) : 20.32x20.32 (cm)	For Child

- **Exposure Time:** Scan time can be adjusted by resolution of 0.1 sec in the range of 0.7 sec to 1.2 sec.

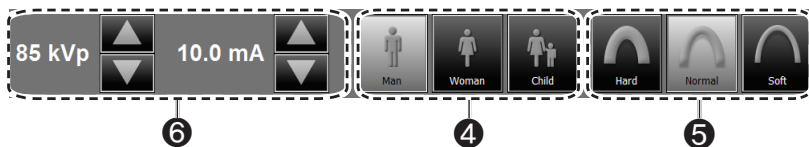4. The patient's gender will be automatically selected according to the patient information registered in EasyDent / EzDent-i. Nevertheless, ensure that this information is correct.

Age Group		VATECH's Standard
	Child	≤ 12
Adult	Man	≥ 13
	Woman	

5. Select X-ray intensity.

NOTE

**X-ray intensity (Hard, Normal, Soft) is according to the operator's decision.**

**Soft ≤ Normal ≤ Hard**

6. A default value for the tube voltage (kVp) and current (mA) will be displayed based on the gender and X-ray intensity. If necessary, further adjustments can be made using the arrows to the right of each number.

7. Click **CONFIRM** to accept the parameters.

When you click the **CONFIRM** button:

- The **READY** button will begin blinking to show that it has been activated.

- The scan time and DAP (Dose Area Product) value are shown on the main display for the intended exposure.

```
Scan Time
12.9 Sec

DAP
21.326724 mGy x Cm^2
```


8. Guide the patient to the equipment.

## 4.2 Positioning the Patient

Follow the procedure below to prepare and position the patient for CEPH imaging.

### Before Positioning the Patient

- Ask the patient to remove all jewelry and metallic objects, such as earrings, hair pins, spectacles, dentures, and orthodontic appliances. These items may cause shadow images which could obscure diagnosis.
- It is strongly recommended that the patient wear a lead apron for protection against any possible scatter radiation.
- Adjust the equipment to the patient's height by using the Vertical Frame up/down button or switch.

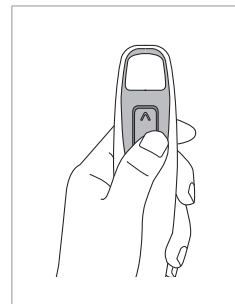**Vertical Frame  
Up/Down  
switch(optional)**

NOTE

**Correct positioning is an important factor in capturing the best possible image.**

CAUTION

**Ensure that the nasal positioner left unfolded, before adjusting the ear rods in the proper direction.**


WARNING

**Ensure that the patient is clear of all moving parts while adjusting the height of the equipment.**

### 4.2.1 Lateral / Full Lateral

Follow the procedure below to position the patient.

1. Turn the CEPH ear rods to **Lateral** mode position as shown in the figure.

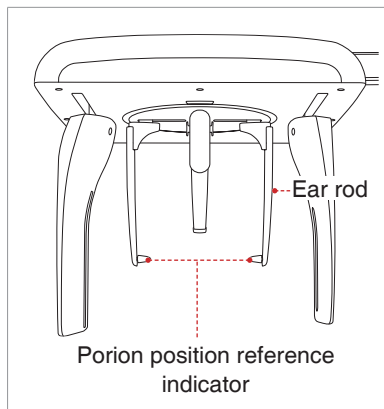2. Widen the distance between the two ear rods.


NOTE

**Porion position reference indicator enables the operator to easily identify the position of the porion on the image.**

3. Guide the patient to the CEPH unit.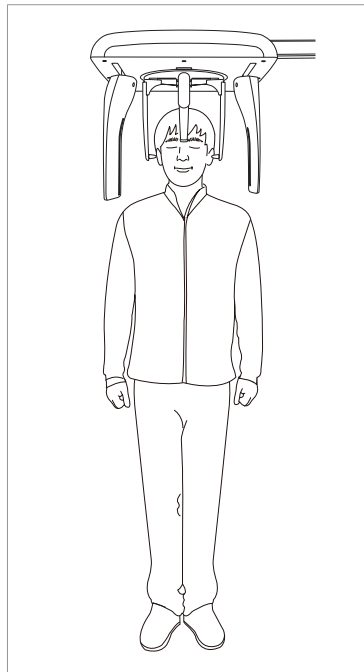


4. Adjust the height of the unit to suit the patient by pressing the column up/down button or switch (optional).

5. Ask the patient to stand upright.

6. Make sure that the patient's shoulders are level and that their neck is relaxed.

After adjusting the height of the column to suit the patient, fit the ear rods along the patient's ear canals and adjust the nasal positioner.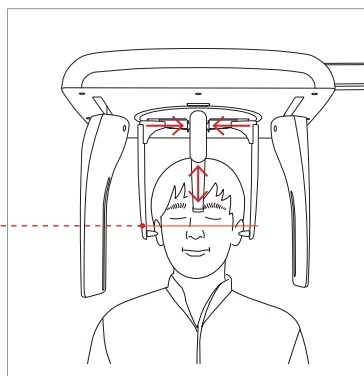

Frankfurt plane

7. The ear rods should fit properly along the patient's ear canals. The patient's Frankfurt Plane should be parallel to the floor.

8. Position the nasal positioner on the patient's Nasion point. The height of nasal positioner can be adjusted.

9. Ask the patient to swallow any saliva in his/her mouth and remain still until X-Ray exposure is complete.

10. Click the **Ready** button after the patient has been properly positioned. No X-Ray will be emitted at this point.

11. Proceed to section **4.3 Initiating X-Ray Exposure**.


## 4.2.2 PA

Follow the procedure below to correctly position the patient.

1. Turn the CEPH ear rods to **PA** mode position as shown in the figure.
2. Widen the distance between the two ear rods.
3. The nasal positioner should be flipped to the side and upward to prevent it from obstructing the image acquisition.

4. Guide the patient to the CEPH unit.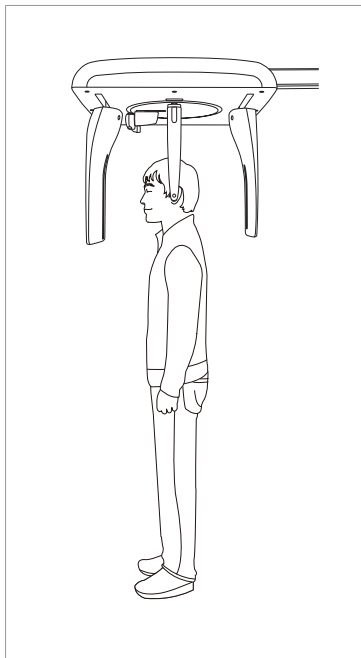

5. Ask the patient to stand upright facing the sensor.


Make sure that the patient's shoulders are level and that his/her neck is relaxed.

6. Adjust the height of the unit to suit the patient by pressing the column up/down button or switch (optional).

CAUTION

After adjusting the column to the patient's height, fit the ear rods along the patient's ear canals.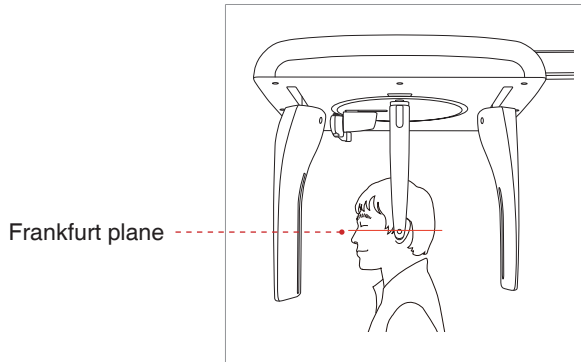

7. The ear rods should fit properly along the patient's ear canals. The patient's Frankfurt Plane should be parallel to the floor.

8. Ask the patient to swallow any saliva in his/her mouth and remain still with his/her mouth closed until X-Ray exposure is complete.

9. Click the **Ready** button after the patient has been positioned. No X-Ray will be emitted at this point.

10. Proceed to section **4.3 Initiating X-Ray Exposure**.

### 4.2.3 SMV


Follow the procedure below to prepare and position the patient.

1. Turn the CEPH ear rods to **SMV** mode position as shown in the figure.

2. Widen the distance between the two ear rods.

3. The nasal positioner should be flipped to the side and up to prevent it from obstructing the image acquisition.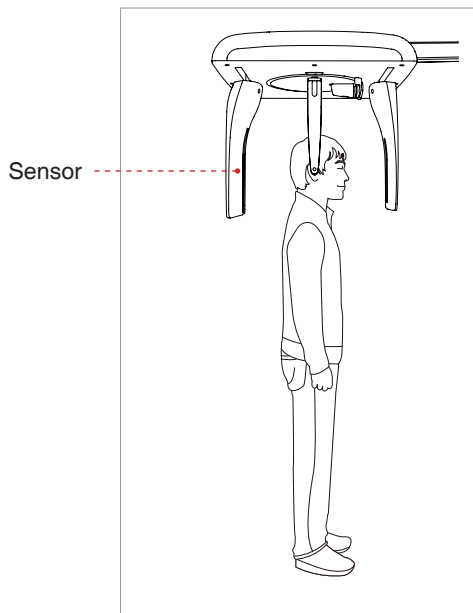

4. Guide the patient to the CEPH unit.

5. Adjust the height of the unit to suit the patient by pressing the column up/down button or switch (optional).


6. Ask the patient to stand upright facing the X-Ray tube.

Make sure that the patient's shoulders are level and that his/her neck is relaxed.

CAUTION

After adjusting the column to suit the height of the patient, fit the ear rods along the patient's ear canals.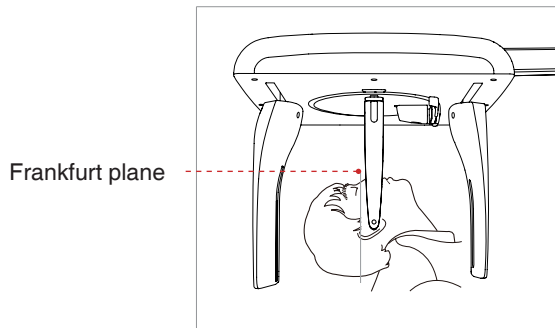

7. Place the ear rods along the patient's ear canals. Make sure that the ear rods are comfortably, yet firmly in place.

8. Gently tilt the patient's head back until his/her Frankfurt plane is perpendicular to the floor, as shown below.

9. Click the **Ready** button after the patient has been properly positioned. No X-Ray will be emitted at this point.

10. Proceed to section **4.3 Initiating X-Ray Exposure**.

#### 4.2.4 Waters View

Follow the procedure below to prepare and position the patient for Waters View mode image acquisition.

1. Turn the CEPH ear rods to **Waters View** mode position as shown in the figure.

2. Widen the distance between the two ear rods.

3. Flip the nasal positioner to the side and up to prevent it from obstructing the image acquisition.

4. Guide the patient to the CEPH unit.


5. Ask the patient to stand upright facing the sensor. Make sure that the patient's shoulders are level and that his/her neck is relaxed.

6. Adjust the height of the unit to suit the patient by pressing the column up/down button or switch (optional).

After adjusting the column to suit the height of the patient, fit the ear rods along the patient's ear canals.

7. Place the ear rods along the patient's ear canals. Make sure that the ear rods are comfortably, yet firmly, in place.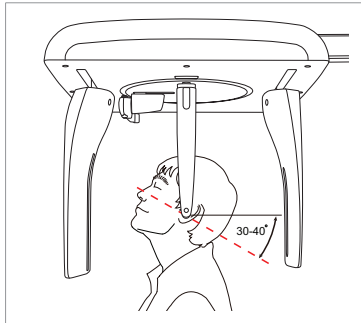


8. Ask the patient to swallow any saliva in his/her mouth and tilt his/her neck back  $30^{\circ}$  -  $40^{\circ}$ , with the mouth closed, until X-Ray exposure is complete.

9. Click the **Ready** button after the patient has been properly positioned. No X-Ray will be emitted at this point.


10. Proceed to section **4.3 Initiating X-Ray Exposure**.

## 4.2.5 CARPUS

### Attaching the CARPUS plate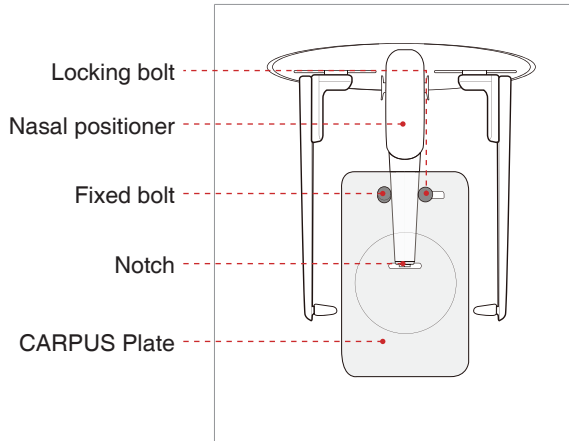

1. Match the notch on the CARPUS plate to the end of nasal positioner.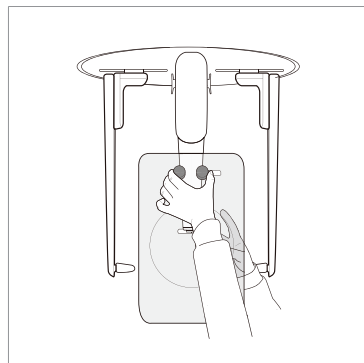


2. Slide the CARPUS plate onto the nasal positioner.

3. Slide the locking bolt on the CARPUS plate towards the nasal positioner and turn it to secure the CARPUS plate in place.

4. Ensure that the CARPUS plate is firmly in place.

## Positioning the Patient

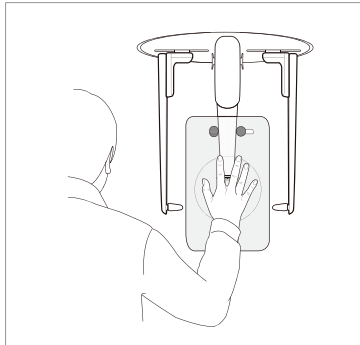1. Ask the patient to put his/her right hand flat on the CARPUS plate. It is important to ensure that the patient does not bend his/her fingers.

2. Have the patient close their eyes and remain still until scanning is complete.

Ask the patient not to block the ruler on the nasal positioner by his / her fingers. It could deteriorate image quality.

3. Click the **Ready** button after the patient has been properly positioned. No X-Ray will be emitted at this point.

4. Proceed to section **4.3 Initiating X-Ray exposure**.

## 4.3 Initiating X-Ray Exposure

The X-Ray exposure method and process is the same for all CEPH modes. The example and images below are taken from an X-Ray performed in **CEPH Lateral** mode.

- **Do not operate the PC during exposure. Failure to comply with this instruction may cause the system to malfunction.**
- **The operator shall observe the X-Ray safety regulations applicable to their area at all times during the operation of this equipment.**


**If an emergency occurs during image acquisition, release the exposure switch to cease X-Ray emission.**

1. Leave the X-Ray room and close the door. The operator must visually monitor the patient at all times during image acquisition.
2. Press and hold down the exposure switch until image acquisition is complete.

Exposure indicator light turns orange

**Orange: Exposure**

X-Ray ON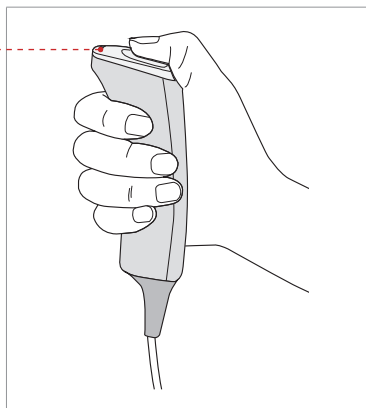

During X-Ray irradiation, ensure that

- The LED lamp on top of the equipment turns orange to indicate X-Ray emission.
- The warning lamp outside the X-Ray room turns on.
- The sound (beep or music: optional) goes off.
- The radiation symbol on the upper left corner of GUI turns yellow to indicate X-Rays are being emitted.

3. Release the exposure switch when “Image capturing is completed” message appears on the screen.
4. The image appears in real time on the imaging GUI.
5. Click the **Save** button to save the acquired image. If automatic save has been selected as a default setting, the image will be saved automatically.

### **After Image Acquisition**


After the image has been acquired, perform the following tasks:

- Fold away the nasal positioner.
- Loosen the ear rod supports and remove them from the patient’s ears.
- Release the patient.

## 5 Acquiring CT Images

### 5.1 Setting the Exposure Parameters

Perform the following procedures to set capture parameters for the specific patient and capture mode.

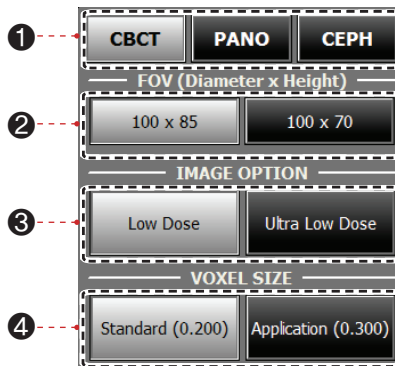1. Click **CBCT**.
2. Select a FOV size.

NOTE


**100 x 85: for Adult, 100 x 70: for Child**

3. Select an image option.
4. Select a voxel size.

NOTE

**MAR(Metal Artifact Reduction) is applied automatically if there are metal objects in the image. MAR may increase image reconstruction time.**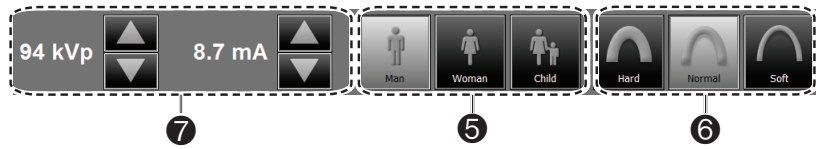

5. The patient's gender will be automatically selected according to the patient information registered in EasyDent / EzDent-i. Nevertheless, ensure that this information is correct.

Age Group		VATECH's Standard
	Child	$\leq 12$
Adult	Man	$\geq 13$
	Woman	

6. Select X-ray intensity.

NOTE

**X-ray intensity (Hard, Normal, Soft) is according to the operator's decision.**  
**Soft  $\leq$  Normal  $\leq$  Hard**

7. A default value for the tube voltage (kVp) and current (mA) will be displayed based on the gender and X-ray intensity. If necessary, further adjustments can be made using the arrows to the right of each number.

8. Click **CONFIRM** to accept these parameters.

NOTE

Please wait for a moment, while the rotating unit moves to its initial scanning position.

When you click **CONFIRM** button:

- The **READY** button will begin blinking to show that it has been activated.

- The Rotating unit will move to its initial scanning position.
- The laser beam(Mid-sagittal) will be activated to facilitate patient alignment.
- The scan time and DAP (Dose Area Product) value are shown on the main display for the intended exposure.

Scan Time  
18.0 Sec

DAP  
646.447615 mGy x Cm<sup>2</sup>



9. Guide the patient to the equipment.

## 5.2 Positioning the Patient

Follow the procedure outlined below to properly position the patient for CT scanning.

### Before Positioning the Patient

- Ask the patient to remove all jewelry and metallic objects, such as earrings, hair pins, spectacles, dentures, and orthodontic appliances. These items can cause shadow images which could obstruct diagnosis.
- It is strongly recommended that the patient wear a lead apron to protect against scatter radiation.
- Adjust the equipment to the patient's height by using the Vertical Frame up/down button or switch.

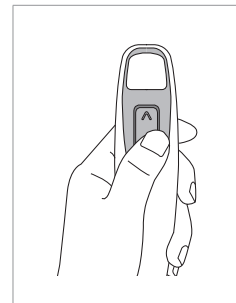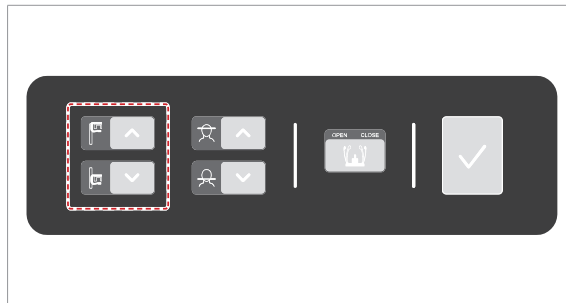**Vertical Frame  
Up/Down  
switch(optional)**

NOTE


Correct positioning is an important factor in capturing the best possible image. Proper positioning will reduce the appearance of the cervical spine in the image.

WARNING

**Be careful not to project the laser beams directly into the patient's eyes as this could severely damage the patient's vision.**

## Positioning the Patient

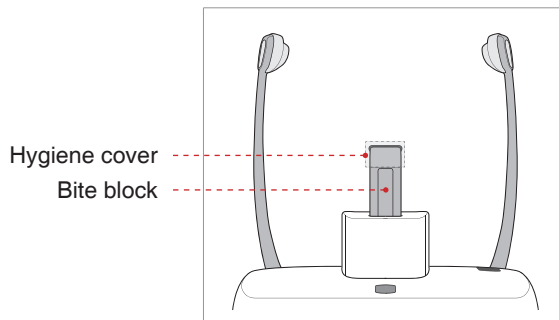1. Insert the bite block into the Chinrest receptacle on the equipment.


2. Place a new hygienic cover over the bite block.

**The hygiene cover for the bite block is intended for single use only. Always replace the hygiene cover for each new patient.**

**Disinfect the Chinrest and bite block using an alcohol-based cleaning solution and wipe away all residue with dry cloth before each new patient.**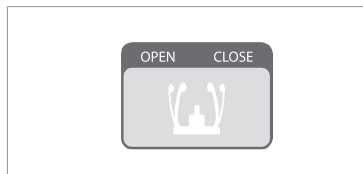

3. Open the temple supports by clicking the Temple Support Wheel Adjustment button.

4. Guide the patient —facing the Chinrest — to the equipment.


5. Adjust the height of the column using the column up/down button or switch (optional) until the patient's chin rests comfortably on the Chinrest.

6. Ask the patient to:

- Stand upright
- Firmly grip the handles on both sides of the unit
- Lean his/her chest lightly against the equipment
- Position his/her feet slightly forward and open stance

Ensure that the patient's shoulders remain level and that his/her neck is relaxed.

The cervical spine should be straight and upright.

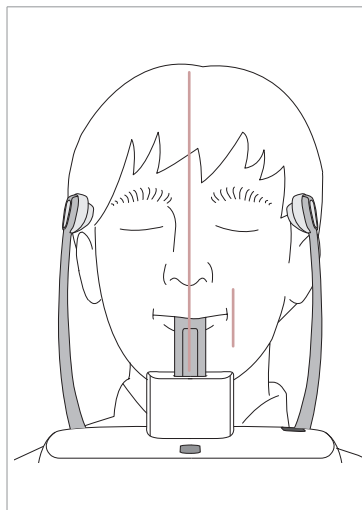7. Have the patient:

- Bite the bite block along its grooves using his/her upper and lower incisors
- Close his/her lips
- Keep his/her tongue pressed against his/her palate
- Close his/her eyes

The patient's occlusion line should be parallel to the floor.

Ask the patient to remain still until scanning is complete.

CAUTION

To acquire the best image possible, ask the patient not to:

- Breathe or swallow saliva during image acquisition
- Move during image acquisition


## Aligning the Laser Beams

**Be careful not to project the laser beams directly into the patient's eyes as this could severely damage the patient's vision.**

Mid-sagittal plane  
laser beam

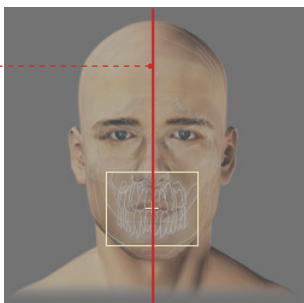
(FOV 100 X 85)

**This is a sample illustration for reference only. Actual FOV may vary from the image shown.**

**Mid-sagittal plane laser beam:** Position the Mid-sagittal plane laser beam at the center of the FOV area.

## Completing Patient Positioning

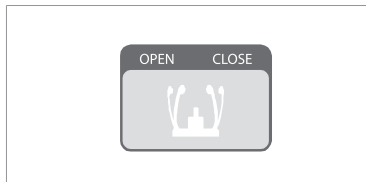1. After confirming patient positioning and beam alignment, adjust the temple supports to fit snugly on both sides of the patient's head using the temple support adjustment button.

2. Click the **Ready** button after the patient has been properly positioned. No X-Ray will be emitted at this point.

3. Proceed to the section **5.4 Initiating X-Ray Exposure**.

## 5.3 Initiating X-Ray Exposure

- **Do not operate the PC during exposure. Failure to comply with this instruction may cause the system to malfunction.**
- **The operator shall observe the X-Ray safety regulations applicable to their area at all times during the operation of this equipment.**


**Direct the patient to not move during the exposure until the Temple support is open.**

1. Leave the X-Ray room and close the door. The operator must visually monitor the patient at all times during image acquisition.
2. Press and hold down the exposure switch until image acquisition is complete.

Exposure indicator light turns orange

**Orange: Exposure**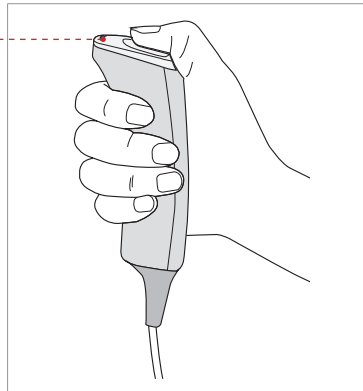

During X-Ray irradiation, ensure that

- The LED lamp on top of the equipment turns orange to indicate X-Ray emission.
- The warning lamp outside the X-Ray room turns on.
- The sound (beep or music: optional) goes off.
- The radiation symbol on the upper left corner of GUI turns yellow to indicate X-Rays are being emitted.

3. Release the exposure switch when "Image capturing is completed" message appears on the screen.
4. The image appears in real time on the imaging GUI.

**If an emergency occurs during image acquisition, release the exposure switch to cease X-Ray emission.**

5. The captured image is automatically reconstructed and converted into a DICOM file. Click the Save button to store the file in the patient database (If automatic save is selected in the program settings, the image will be saved automatically).

Once acquisition is complete, the image is automatically transferred to **EasyDent / EzDent-i**.

NOTE

For more details about 2D or 3D viewer, refer to **EasyDent / EzDent-i**, and **Ez3D plus / Ez3D-i** user manual.

### After Image Acquisition

Perform the following tasks after acquiring a CT image:

- Loosen the temple supports and release the patient.
- Remove the hygiene barrier from the bite block.
- Press Return to bring the Rotating Unit back to its initial position.

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The 'Add Patient' dialog box contains the following fields and controls:

- Chart No.: 20LL1229\_180547 (with an 'Auto' button and a checked 'Auto No.' checkbox)
- \* First Name: [Empty text box]
- \* Last Name: [Empty text box]
- Social ID: [Empty text box]
- Birthday: 2011 (dropdown), 1 (dropdown), 1 (dropdown)
- Gender: Male (dropdown), Treatment: Treatment (dropdown)
- Address1: [Empty text box]
- Address2: [Empty text box]
- E-mail: [Empty text box] @ [Empty text box]
- Tel: [Empty text box] Mobile: [Empty text box]
- Doctor: [Empty dropdown menu]
- Buttons: Add, Cancel

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- Chart No.: 20111223\_180547 (with an 'Auto' button and a checked 'Auto No.' checkbox)
- \* First Name: [Empty text box]
- \* Last Name: [Empty text box]
- Social ID: [Empty text box]
- Brthday: 2011 (dropdown), 1 (dropdown), 1 (dropdown)
- Gender: Male (dropdown), Treatment: Treatment (dropdown)
- Address1: [Empty text box]
- Address2: [Empty text box]
- E-mail: [Empty text box]
- Te: [Empty text box], Mobile: [Empty text box]
- Doctor: [Empty dropdown menu]
- Buttons: Add, Cancel

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**Copyright © 2015 par VATECH**
**Copyright © 2015 VATECH**