# Sting

**ISTRUZIONI PER L'USO  
OPERATING INSTRUCTIONS  
MODE D'EMPLOI  
MANUAL DE INSTRUCCIONES**

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The manufacturer declines all and any responsibility for possible inaccuracies due to typing errors contained in this manual and reserves the right to modify his products without changing their main characteristics, if he considers such modifications necessary or useful.

**To be preserved for future consultation.**

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All polyurethane parts produced by Tecnodent are:

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The chair complies with:  
Medical Device Directive 93/42/EEC.

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# INDEX

1.	Introduction	22
1.1.	Definitions.	22
1.2.	General warning.	22
1.3.	Purpose of the information contained in this document.	22
1.4.	Document keeping.	23
1.5.	Limitation of responsibility.	23
1.6.	Transport and storage.	23
1.7.	Packing disposal.	23
1.8.	End-of-life equipment disposal information.	23
2.	Equipment description	24
2.1.	Intended use.	24
2.2.	General features.	24
2.3.	Warranty.	24
2.4.	Identification and labelling.	24
2.5.	Technical specifications.	25
3.	Installation	27
3.1.	Unpacking.	27
3.2.	Supply status.	27
3.3.	Handling.	27
3.4.	Installation.	27
4.	Using the product	29
4.1.	Typical use.	29
4.2.	Controls.	29
4.3.	Chair operations.	30
5.	Maintenance	32
5.1.	General informations.	32
5.2.	Cleaning.	32
5.3.	Main fuses replacement.	32
5.4.	Upholsteries replacement.	32
6.	Troubleshooting.	33
7.	Accessories	35
7.1.	Headrests.	35
7.2.	Armrests.	36
7.3.	Programs.	36
7.4.	Trendelenburg position.	36
7.5.	Remote control.	36

# 1. INTRODUCTION

## 1.1. DEFINITIONS.

The following graphical and linguistic definitions have been used in this manual.

### NOTE.

Important information, which has to be underlined in relation to the text.

### CAUTION.

This message can appear before some procedure descriptions. Non-compliance may cause damage to the equipment.

### WARNING.

This message can appear before some procedure descriptions. Non-compliance may cause damage to the equipment and to the operator.

#### **Double function control(s).**

Foot control switch which activates different chair functions, depending on how it is pressed (i.e. press and hold: manual chair movement; tap: automatic movement of a pre-programmed working position).

#### **Synchronized movement.**

Simultaneous and interdependent variation of the position of two chair parts. It may occur both during a manual or an automatic movement (i.e. chair lifting combined with simultaneous backrest tilting).

#### **Compensated movement.**

Position variation of two chair parts acted by a motion system which takes into consideration ergonomical needs (i.e. backrest leaning movement).

#### **PCB.**

Printed circuit board.

#### **CN.**

Electric connector.

#### **PWR.**

Power supply.

#### **LED.**

Light-emitting diode.

## 1.2. GENERAL WARNING.

On request, Tecnodent will make the components part list, descriptions, calibration instructions or other information available, which will assist the user's technical personnel to carry out repairs of those parts of the chair which are designed by the manufacturer as repairable.

### WARNING.

**The manufacturer will accept responsibility for the security, reliability and performance of the equipment on the following conditions:**

- **Installation and any repairs or modifications are carried out by authorized personnel only;**
- **The electrical system of the premises complies with all regulations currently in force at the time of installation;**
- **The equipment is used in accordance with the operating instructions.**

### NOTE.

Please note that in accordance with Art. 14 of EEC Directive 85/374 "Liability for damages arising from defective products" implemented in Italy by Decreto del Presidente della Repubblica 24 maggio 1988, n. 224; "Right to compensation ceases ten years after the day the manufacturer or the importer within the EU Nations first marketed the product, which is object of the claim".

## 1.3. PURPOSE OF THE INFORMATION CONTAINED IN THIS DOCUMENT.

This manual shows the Sting Dental Chair and contains information regarding its functions, performances, servicing, troubleshooting and corresponding solutions.

This manual is intended for the final user, the professional that uses the equipment to perform his job.

## 1.4. DOCUMENT KEEPING.

This manual is part of the equipment and must be conserved until its disposal. It should be stored in a protected location, but near the equipment for prompt usage if necessary; the user, in case of loss, must ask a spare copy.

## 1.5. LIMITATION OF RESPONSIBILITY.

The Manufacturer must be relieved of any responsibility in the following cases:

- Improper use or usage by not specifically trained personnel;
- Use while neglecting the applicable standards;
- Incorrect installation, when not specifically included in the supply contract;
- Power supply failure or defects;
- Lack of servicing;
- Unauthorized modification or interventions;
- Use of non-original spare parts, or not model-specific;
- Neglecting the instructions for use, even if partial;
- Exceptional events.

## 1.6. TRANSPORT AND STORAGE.

While packed for transport or storage, the chair can be exposed to environmental conditions which do not lie outside the following ranges, for a period not exceeding 10 weeks:

	Min	Max
Environment temperature range	-5°C	+40°C
Relative humidity range (condensation included)	10%	90%
Atmospheric pressure range	500 hPa (mbar)	1050 hPa (mbar)

## 1.7. PACKING DISPOSAL.

The materials mentioned below, used for packing, respect the environment and are 100% recyclable:

- Wooden pallet with fumigation treatment;
- Cardboard box;
- Polythene film or with air bubbles.

Please deliver the packing to an authorized dump collecting these materials.

## 1.8. END-OF-LIFE EQUIPMENT DISPOSAL INFORMATION.

According to 2002/95/EEC and 2003/108/EEC Directives, concerning the reduction of hazardous substances in electric and electronic equipment, including its waste disposal, Tecnodent specifies the following notes.

- This symbol, placed in the identification label, indicates that the equipment collection and disposal, must be performed separately from other kind of wastes.
- The separate collection of end-of-life equipment is organized and managed by the manufacturer. The owner who would like to dispose this equipment should contact the manufacturer or one of its representatives to allow separate collection of this end-of-life equipment.
- The proper separate collection for recycling, processing and environmentally friendly disposal helps avoiding potential adverse effects on human health, and promotes reuse and recycling of the equipment materials.
- The illegal disposal of the product by the owner is subject to the penalties determined by the laws in force.
- When the chair reaches the end of its working life, it is necessary to definitely put it out of service, by disconnecting the plug from the socket and cutting the power supply cable.

## 2. EQUIPMENT DESCRIPTION

### 2.1. INTENDED USE.

The product described by this manual is a dental patient chair.

The equipment is designed for lifting and holding the patient in a proper working position during dental diagnosis or therapy. It can be used by dental field specialists in private clinics, hospitals, specialized surgeries.

This equipment needs that the premises electrical system is conforming to the specification described in the "installation" section. Usage near equipment monitoring the patient's vital signs is discouraged.

#### WARNING.

**The equipment was developed and manufactured only for dental use. No other kind of use is permitted. It is furthermore not permitted to perform any modification of the equipment or its parts, without a previous written authorization of the manufacturer.**

### 2.2. GENERAL FEATURES.

#### 2.2.1. Basic version.

A dental patient chair model Sting includes the following features:

- Completely electromechanical seat lifting/lowering and backrest tilting movements;
- Compensated backrest tilting, up to the horizontal position;
- Anatomical backrest;
- Multi-articulated headrest;
- Complete symmetry, for right-handed and/or left-handed professionals;
- Foot control on the base;
- Security STOPS on the backrest and on the lifting arm;
- Return to "zero" position automatic program;
- Rinse position automatic program;
- Low voltage controls: 5 V.

#### 2.2.2. Accessories.

The chair can also be equipped with different accessories, depending on the working needs of the professional, as well as the comfort needs of the patient to be treated. In detail, the following optional are available:

- ELLE MOBILE or ELLE-2 MOBILE type headrest;
- Magnetic pillow model C95;
- Magnetic pillow model C2002;
- Right or left armrest(s);
- Device for programming three different working positions;
- Trendelenburg movement;
- Infra-red remote control;
- Dental unit stabilizer.

The operation and the performance of these accessories are described in chapter 7.

### 2.3. WARRANTY.

The Warranty Certificate is delivered together with the equipment. If such certificate would not be sent, it must be immediately requested to the dealer. The Warranty Certificate must be completely filled in and sent back to the manufacturer within 8 days from the date of delivery of the equipment. If this procedure is not observed, the warranty conditions are not valid and the servicing technicians will be compelled to completely charge any needed repairing.

#### NOTE.

Upholstery damage caused by contact with clothes or clothing accessories (ex: belts) having metal, sharp, pointed or similar parts will not be replaced under warranty.

### 2.4. IDENTIFICATION AND LABELLING.

The chair identification label, where you can find the chair model, the serial number and other information, is placed in the base of the chair under the seat. Figure 1 shows the label position and a facsimile.

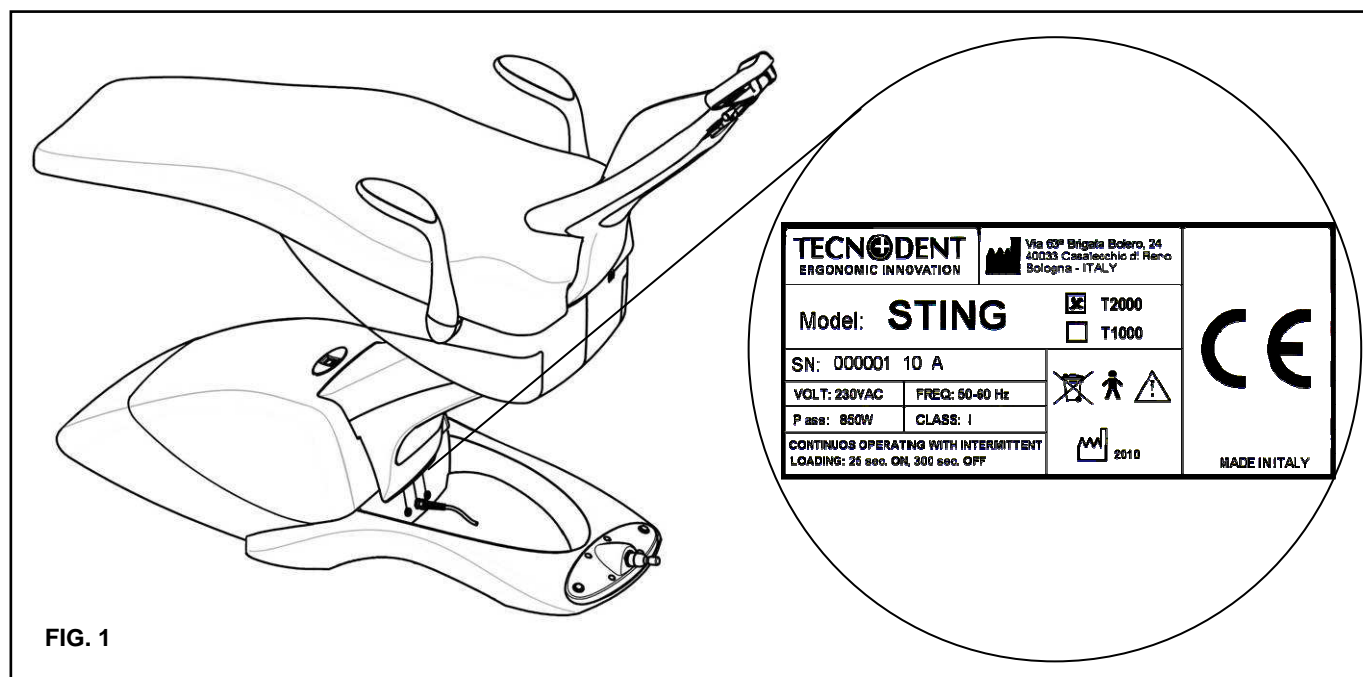
FIG. 1

## 2.5. TECHNICAL SPECIFICATIONS.

### 2.5.1. Sector-specific standard compliance.

The dental patient chair model Sting complies with Medical Device Directive 93/42/EEC.

Specifically, design and manufacture of the equipment comply with the following international standards:

IEC 60601-1:1996	=	EN 60601-1:1996	=	CEI EN 60601-1:1998
IEC 60601-1-2:2001	=	EN 60601-1-2:2001	=	CEI EN 60601-1-2:2003
ISO 6875:1995	=	EN ISO 6875:1996	=	UNI EN ISO 6875:1998

### 2.5.2. Technical data.

Power supply – voltage	V	230 AC
Power supply – frequency	Hz	50 – 60
Power supply – input power	W	700
Main fuses		2 x T 6,3A; 250V
Controls voltage	V	5
External controls interface		Yes
Insulation class		I
Applied parts type		B
Medical device risk class		I
Max. height	mm	820
Min. height	mm	420
Width	mm	644
Length (with completely elongated headrest)	mm	2070
Gross weight	Kg	110
Net weight	Kg	80
Packing dimensions	mm	1430x700x970 (h)
Maximum lifting capability (patient + unit)	Kg	135 + 75
Minimum space required for installation	m	3 x 2

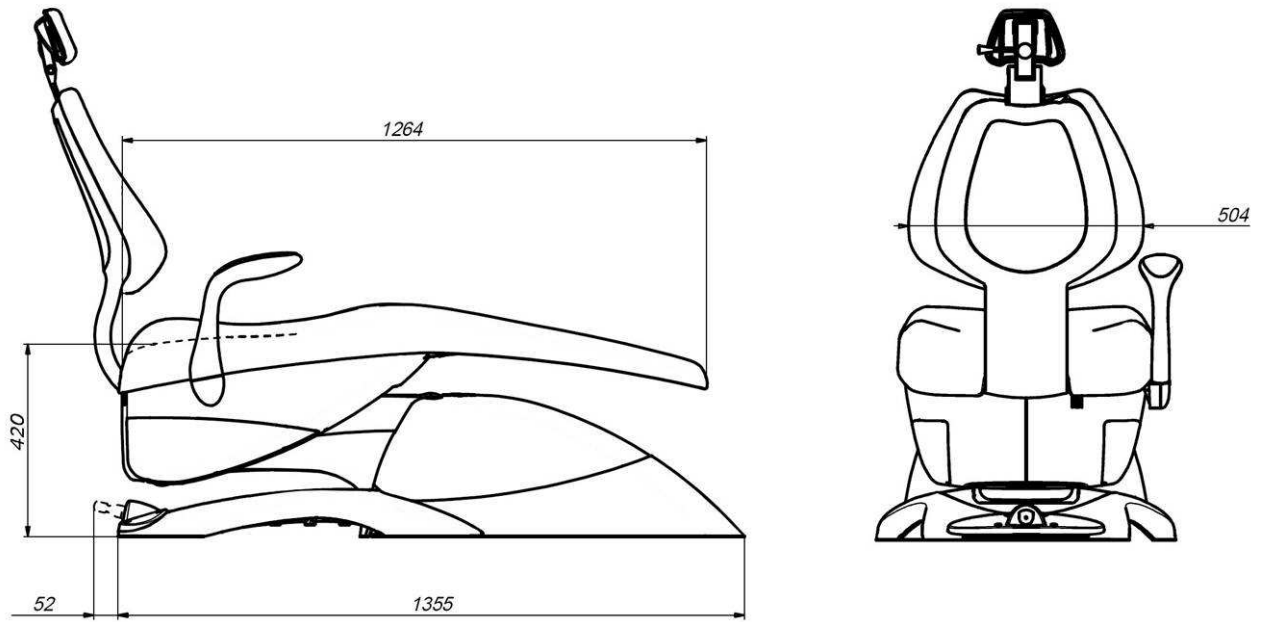
FIG. 2

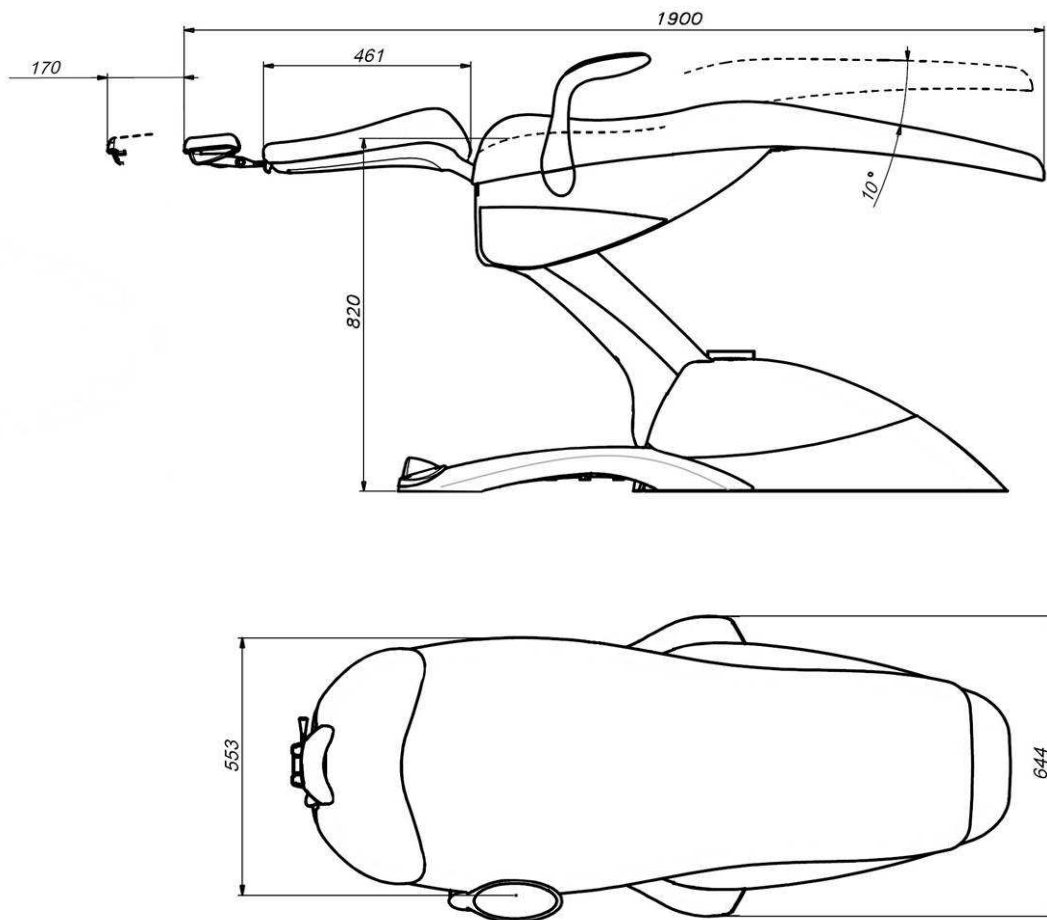
FIG. 3

## 3. INSTALLATION

### 3.1. UNPACKING.

To remove the cardboard box, it is necessary to cut the straps and to unscrew the screws located in the lower part of the box.

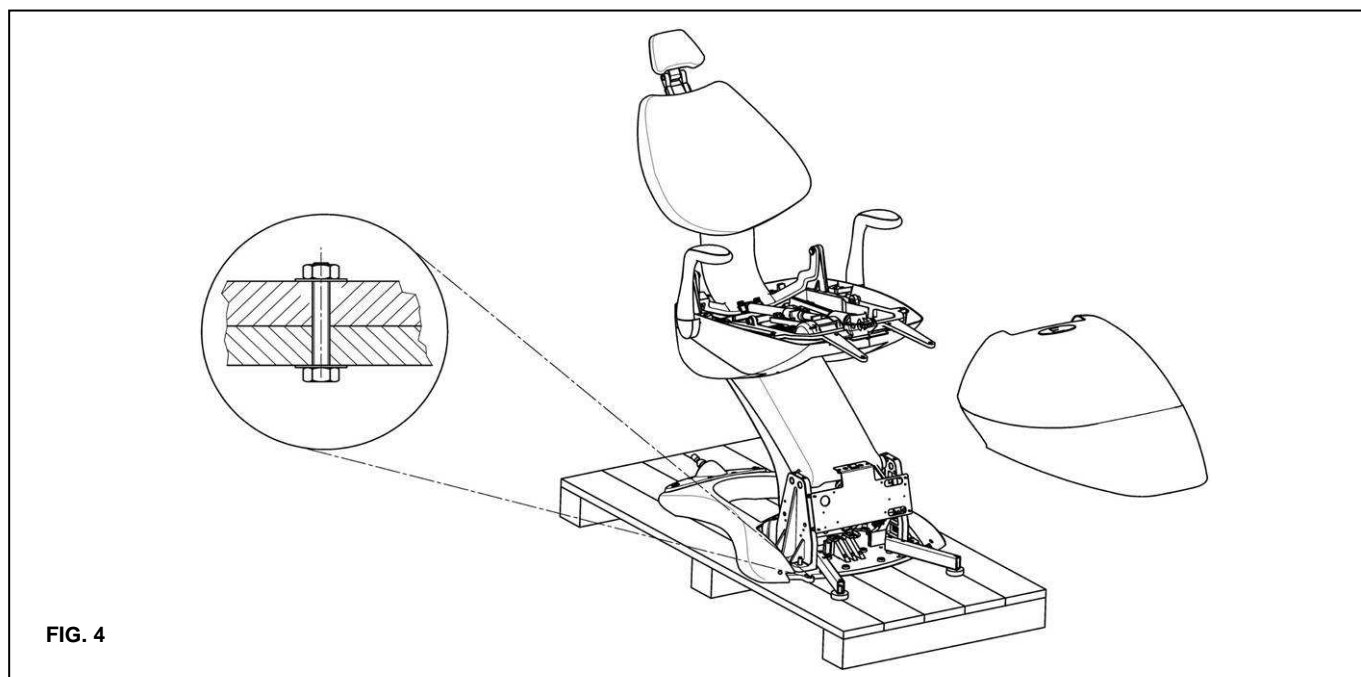
FIG. 4

### 3.2. SUPPLY STATUS.

Inside the packaging you will find the following items:

- A Sting chair;
- An operating instruction manual;
- A warranty certificate;
- A bottle of fabric cleaner;

In case of mistake or missing parts, contact the dealer to manage the problem.

### 3.3. HANDLING.

To remove the chair from the wooden pallet, first take the two caps off, then unscrew the two screws which fix the chair base to the pallet (see figure 4).

To move the chair from the wooden pallet employ two people.

To move the chair it is necessary to use the dedicated handles, provided to the installation service on request.

### 3.4. INSTALLATION.

#### 3.4.1. Typical installation.

Proceed as follows:

1. Be sure that the installation site conforms to local regulatory technical requirements for rooms suitable for medical use or specialized surgery;
2. Be sure that the electrical system of the cabinet has a good earthing and a residual-current device system;
3. Be sure that the main supply of the electrical wiring of the cabinet corresponds to the one of the identification label;
4. Check that the fuses, located at the front side of the chair near to the base, are correctly fixed, because during transport they could have become loose;
5. Insert the plug into the socket;
6. Activate the chair switching the main switch on. Now the chair is ready to work. In case of non operation of the chair, refer to section 6 about trouble-shooting;
7. If needed, drive the chair to a comfortable height and assemble, if provided, the unit bracket and the bracket itself, following the instructions of the manufacturer.

#### NOTE.

Equipment is intended neither for use in potentially explosive environment, nor in rooms not protected against weather conditions. Not intended for use in altered atmospheres.

### 3.4.2. Position choice and stability.

To ensure a good chair installation, please verify that the floor in the location where the product will be installed is stable and plane: if necessary, a damaged or unstable floor should be repaired.

The chair is designed to be stable without floor fixing. To reach that aim, it is provided with two levelling pads (Figure 5, location A and B) to level the base even on slightly odd surfaces.

**NOTE.**

For packaging reasons, the leveling rods are furnished dismounted. It is necessary to assemble them using the holes 1, 2, 3 and 4 shown in figure 5.

**WARNING.**

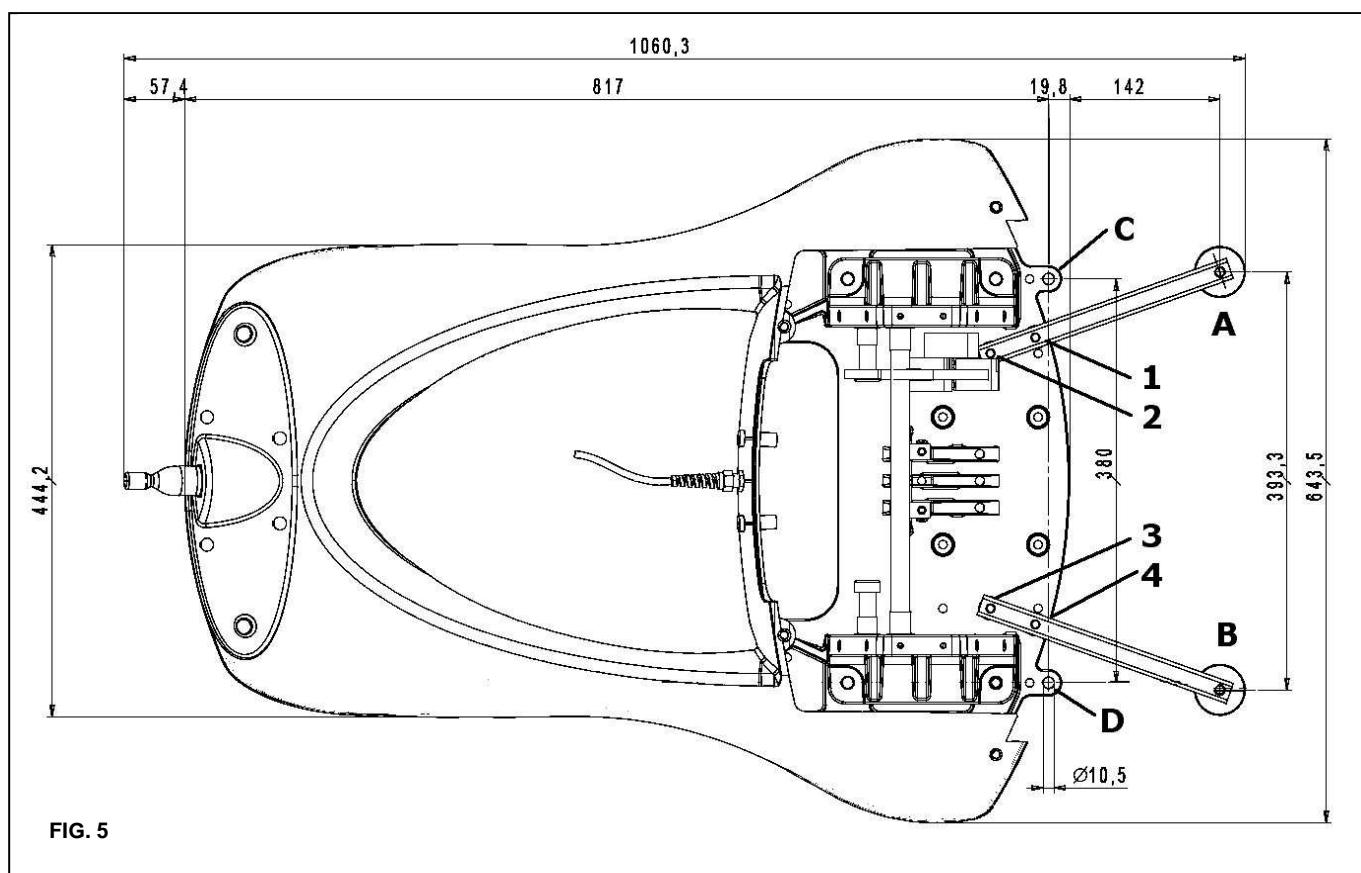
Adjust the levelling pads in such a way that the central part of the base (under the lifting motor) is in contact with the floor, or it is lifted less than 0.5 mm.

To ensure the same stability while a unit is mounted on the chair, two options are available:

- Fixing of the chair on the floor (using C e D holes shown in figure 5);
- Purchasing of the chair with the stabilizing plate.

For more information, ask the installer.

**WARNING.**

Tecnodent declines all and any responsibility for damage caused by the non-compliance with the above instructions.

## 4. USING THE PRODUCT

The advanced technology applied to the production of this apparatus has allowed the design of a dental patient chair with multiple functions and performances, all of them easily usable: all regulations and functions of the chair are coordinated, processed and stored in a "data bank", by a microprocessor inside the chair. The dental patient chair Sting is designed to be easily controlled by foot controls, allowing the professional's hands to be completely free to operate in the patient's mouth.

### WARNING.

Before using the information contained in this section, make sure that the chair is correctly installed, by the service technician.

### CAUTION.

Use in a clean place (specialized surgery, clinic) and avoid accidental spilling or falling of liquid substances on the product.

### CAUTION.

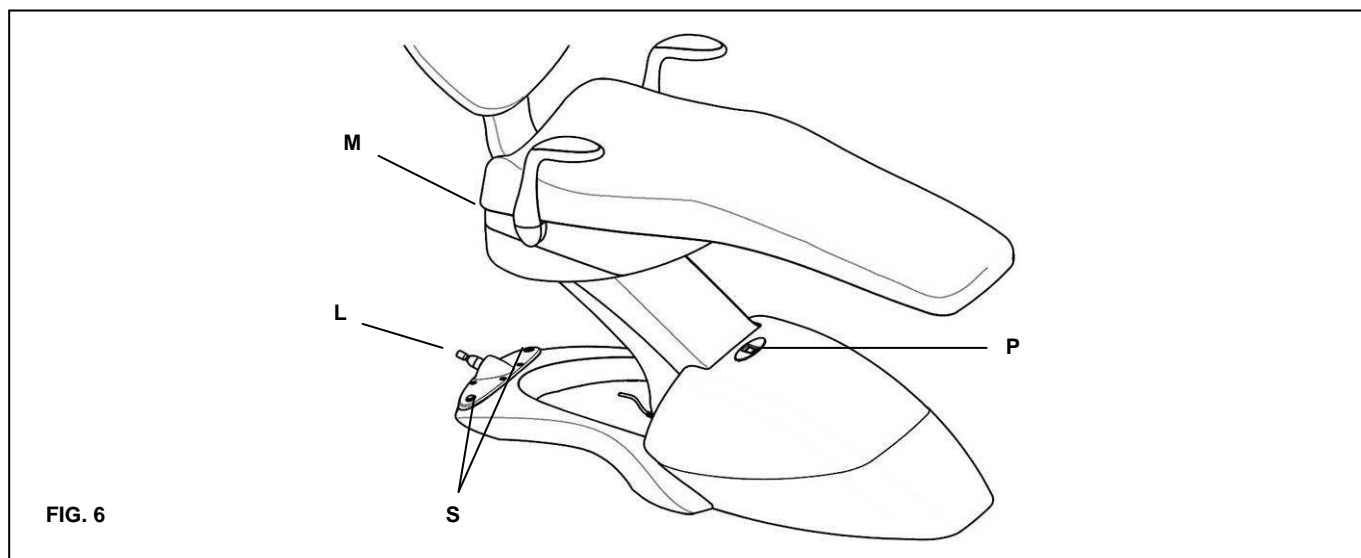
In case of a product malfunction that is not described in this documentation, put equipment out of service (shut it down, unplug main supply, mark the equipment as "out of order" or similar) and call the technical service.

### 4.1. TYPICAL USE.

To use the product, the operator must be near the chair, typically on the back of the chair or sideways, while acting on the controls. Please pay due attention during chair use, because the presence of automatic movements could cause hazardous situations, even if moderate level ones.

The chair is provided with several emergency stop systems, i.e. in case of presence of obstacles during the movements, or if the operator checks any unsuitable situation.

### 4.2. CONTROLS.

#### 4.2.1. Sting without programs.

The controls are shown in figure 6.

- P Main switch (Power on/off).
- L Foot control, placed on the base. It controls raising, lowering and backrest tilt manual movements of the chair and the automatic return to "zero" position.
- S Controls the automatic movement of rinse position, in order to allow the patient's mouth wash and the return to the last working position.

### 4.2.2. Sting with programs.

The controls are shown in figure 6.

- P Main switch (Power on/off).
- L Double function foot controls, placed on the base. They control the raising, lowering and backrest tilt manual movements of the chair, as well as the 3 programmable working positions and the automatic reset to "zero" position.
- S Controls the automatic movement of rinse position, in order to allow the patient's mouth wash and the return to the last working position.
- M Stores the programmed working positions.

## 4.3. CHAIR OPERATIONS.

### 4.3.1. Sting without programs.

#### “L” JOYSTICK OPERATION

- While keeping the joystick downward the chair lowers.
- While keeping the joystick upward the chair lifts.
- While keeping the joystick to the right of the chair, the backrest moves to the vertical position.
- While keeping the joystick to the left of the chair, the backrest moves to the supine position.
- While tapping the joystick downward the chair moves automatically to the reset "zero" position.

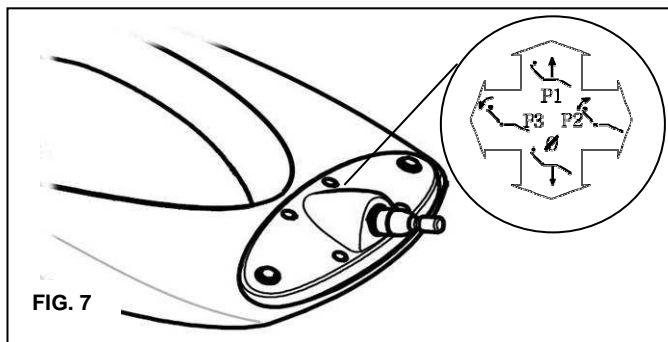
FIG. 7

#### “S” BUTTON OPERATION

While pushing "S", the rinse position function is activated; the chair backrest moves up to vertical position to allow the mouth-wash.

While pushing "S" again, the chair backrest moves back to its last position.

#### NOTE.

Any switch, if pushed during an automatic movement, works like a security STOP.

#### NOTE.

Any switch, if pushed during the automatic reset to "zero" position movement, works like a security STOP.

### 4.3.2. Sting with programs.

#### “L” JOYSTICK OPERATION

- While keeping the joystick downward the chair lowers.
- While keeping the joystick upward the chair lifts.
- While keeping the joystick to the right of the chair, the backrest moves to the vertical position.
- While keeping the joystick to the left of the chair, the backrest moves to the supine position.
- While tapping the joystick upward, left or right, the chair moves automatically to the programmed working positions.
- While tapping the joystick downward the chair moves automatically to the reset "zero" position.

#### “S” BUTTON OPERATION

While pushing "S" the rinse position function is activated; the chair backrest moves up to vertical position to allow the mouth-wash.

While pushing "S" again the chair backrest moves back to its last position.

#### NOTE.

Any switch, if pushed during an automatic movement, works like a security STOP.

#### NOTE.

Any switch, if pushed during the automatic reset to "zero" position movement, works like a security STOP.

**PROGRAMMING PROCEDURE**

This function is only available on chairs equipped with such feature.

1. Let the chair reach the automatic reset to "zero" position, by tapping the "0" button in the foot control.
2. Let the chair reach the desired position, using the foot control (see paragraph 4.3).
3. While keeping the M button pressed, tap the joystick upward (position "1"), right (position "2") or left (position "3") or the buttons "P1", "P2" and "P3" on the remote control, according to the program number you have selected . A sound-signal (BEEP) confirms that the working position has been memorized. Any time you tap the joystick upward, right, left again or the buttons "P1", "P2" and "P3" on the remote control, the chair moves automatically to the programmed working position you have selected.

**NOTE.**

To avoid losing the programmed position, it is recommended to regularly set the chair to its "zero" position, by using the "0" function at any patient change.

## 5. MAINTENANCE

### 5.1. GENERAL INFORMATIONS.

Sting chairs were conceived to not require any servicing during their working life. Consequently, it is not required to perform any adjustment of the equipment. If the chair will not be used for some time, it is recommended to cover it with a cloth, to switch OFF the main switch, and to disconnect the plug from the socket.

### 5.2. CLEANING.

For a better and longer life of the equipment, it is necessary to perform an accurate, methodical and periodical general cleaning of the chair. It is recommended to proceed as follows.

#### 5.2.1. Upholsteries.

The fabric that covers the chair upholstery has to be cleaned using the liquid contained in the bottle kit supplied with the chair, using a soft cloth to avoid any surface scrape and to guarantee a better elasticity and smoothness of the surface.

#### NOTE.

It is recommended to order a new bottle of cleaning product, as soon as it is finished, to the closest servicing centre.

#### 5.2.2. Plastic or metal parts.

The polyurethane parts, as well as the metal parts (painted and not) have to be cleaned with a soft cloth dipped in water and soap.

#### CAUTION.

It is recommended to avoid the use of any detergent or strong abrasive agent, even to remove "difficult" stains.

### 5.3. MAIN FUSES REPLACEMENT.

The Sting chair is protected against electrical power overrating by two fuses, placed as shown in figure 8. To replace them, proceed as follows:

1. Turn off the chair's main switch;
2. Disconnect the plug from the socket;
3. Unscrew the fuse holder cap, by using a crosshead screw driver of medium size.
4. To re-install the fuses, follow the inverse sequence of the above steps.

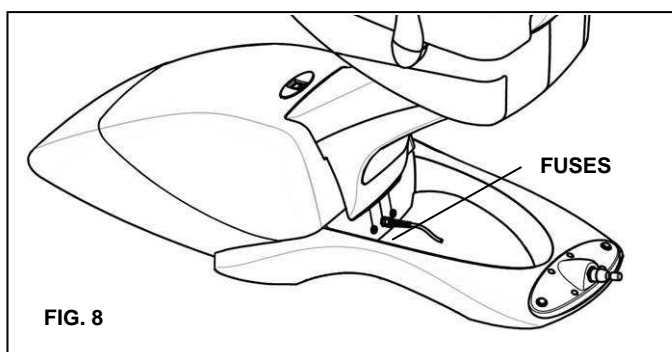
FIG. 8

#### WARNING.

Fuses must be replaced by other fuses having the same nominal characteristics. Table "Technical data" in paragraph 2.5.2 contains the relative specifications.

### 5.4. UPHOLSTERIES REPLACEMENT.

To replace the chair upholsteries, proceed as follows and as shown in figure 9.

#### 5.4.1. Seat.

1. Unscrew the screws 1 and 2 from the seat support.
2. Remove the upholstery, by pulling it up from its support, and uncoupling it from the plastic pins that keep it fixed.
3. To re-install the seat upholstery, follow the inverse sequence of the above steps.

#### 5.4.2. Backrest.

1. The backrest upholstery is mounted by pressure with four special plastic caps. To remove it from its counterpart it is necessary to pull it gently away from its counterpart.
2. To re-install the seat upholstery, follow the inverse sequence of the above steps.

#### 5.4.3. Headrest.

To replace headrest upholstery, see paragraph 7.1 for reference.

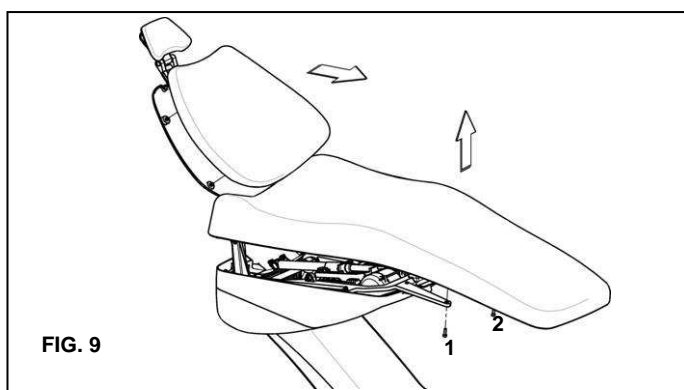
FIG. 9

## 6. TROUBLESHOOTING.

The steps to detect a faulty condition on the Sting chair are described in table-form. Each table reports the title of the troubleshooting procedure, and three columns whose content can help to detect and solve the problem. Below is an example:

<b>Procedure Name</b>		
<b>Problem</b>	<b>Test Procedure</b>	<b>Corrective Action(s)</b>
Dental patient chair problem description.	If the answer to the proposed question is "YES" look at the question written in the next box below; if the answer to the question is "NO" take all the corrective action(s) described in the right box before moving to the next question.	Action(s) to be carried out in order to fulfil the requirements of the question asked by the test procedure at the side. Only after all actions have been carried out with positive result it is possible to move to the next test procedure.

The table-form takes into consideration only electrical or electronic faulty conditions as mechanical faults are easily detectable by a simple visual inspection. In order to execute a visual inspection it is sufficient to remove the upholstery parts, as described in paragraph 5.4.

<b>Main supply control procedure</b>		
No chair movement.	Is the supply voltage correctly rated?	If an OHM-meter or a similar testing device is not available, plug any other apparatus (i.e. a lamp) into the socket and verify the presence of the power supply.
	Is the power cord plugged in?	Plug the cord into the socket.
	Is the green light of the main switch on?	Turn the main switch on. After having unplugged the chair, check the main fuses. If necessary, replace them as described in the servicing manual. Otherwise call the technical service.
	Does the PCB emit a "BEEP" sound when the main switch is turned on?	The PCB has probably failed. Call the technical service.
	The problem is not related to the chair main supply group. Continue checking the chair movements one by one.	

<b>Chair lifting movement control procedure</b>		
The chair does not lift.	Is there too much weight on the chair?	Apply the chair movement testing procedure.
	Remove the excessive weight. Look at the chair operating instruction manual to determine the maximum chair lifting capability	

<b>Backrest tilting movement control procedure</b>		
The chair backrest does not tilt.	Is there an object that collides with the chair, activating the security device and stopping the backrest tilt movement?	Apply the chair movement testing procedure.
	Remove the object that collides with the chair.	

<b>Backrest raising movement control procedure</b>		
The chair backrest does not raise.	Was the chair movement testing procedure duly applied?	Apply the chair movement testing procedure.
	Call the technical service.	

<b>Chair movement testing procedure</b>		
The chair does not move at all, even if the main switch is switched on and the mains group is working correctly.	Do you hear a "CLICK" sound when you try to activate the controls?	If no "CLICK" sound is emitted probably the PCB has failed. Call the servicing technicians.
	Does the motor hum?	The motor may have over-heated and the thermal security switch may have turned on. Wait for about 15 minutes, allowing the motor to cool down and operate the motor again. If the chair does not move or the motor over heats immediately after, call the servicing technicians.
	It is impossible to determine the cause of the failure unless further and more complex testing is applied. Call the servicing technicians.	

<b>Chair programming control procedure (only for chairs with programs)</b>		
The chair moves properly in all manual movements, but it is impossible to store working position programs.	Was the programming procedure properly applied? (see paragraph 4.3.2 for reference)	Apply the procedure as quoted in paragraph 4.3.2.
	Did you hear a "BEEP" sound when the programming procedure was completed, to confirm the programs' saving?	It is impossible to determine the cause of the failure unless further and more complex tests are performed. Call the technical service.
	The PCB has probably failed. Call the technical service.	

## 7. ACCESSORIES

### 7.1. HEADRESTS.

The Sting chair can be provided with different kinds of headrests.

#### 7.1.1. Headrest type "UNI tappezzato".

This headrest allows the optimum positioning of the patient's head, according to the required treatment, thanks to its double articulating mechanism and the possibility to adjust its height (see Figure- 10).

##### OPERATION

- To regulate the double articulation, turn clockwise the "R" handle to unlock the mechanism. To lock the mechanism in the desired position, turn anti-clockwise the "R" handle
- By pushing the "H" lever, the mechanism is released, allowing the headrest to freely move to adjust its height. Releasing the lever, the headrest remains in the selected position.

##### MAINTENANCE

The headrest upholstery can be easily replaced, by simply pulling and separating it from its counterpart.

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These headrests allows the positioning of the patient's head by adjusting its elongation as well as its inclination angle, as shown on figure 11.

##### OPERATION

- By pushing the headrest from its back part towards the patient's head, can be selected the forward position. The backward position can be reached by pushing the headrest from its frontal part, while simultaneously pushing the release button "T".
- By pushing the "H" lever, the mechanism is released, allowing the headrest to freely move to adjust its height. Releasing the lever, the headrest remains in the selected position.

##### MAINTENANCE

The headrest upholstery can be easily replaced, by simply pulling and separating it from its counterpart.

#### 7.1.3. "C95" or "C2002" magnetic pillow.

The magnetic pillow, as shown in figure 12, can be used to hold the patient's head during a specific treatment. It can be attached to the surface of the ELLE MOBILE, ELLE-2 MOBILE headrest, or in the backrest upper section, using it in the cases where the patient's height is not enough to reach the headrest.

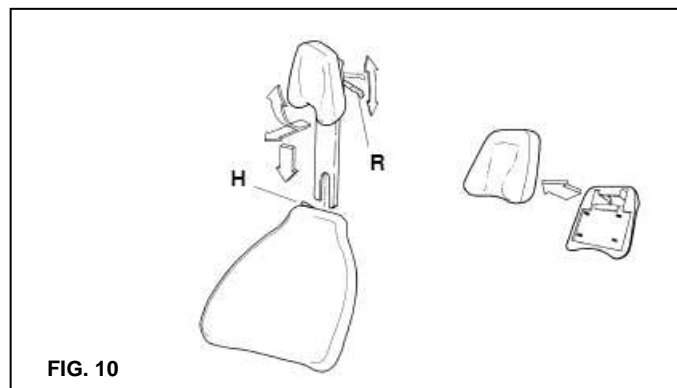
FIG. 10

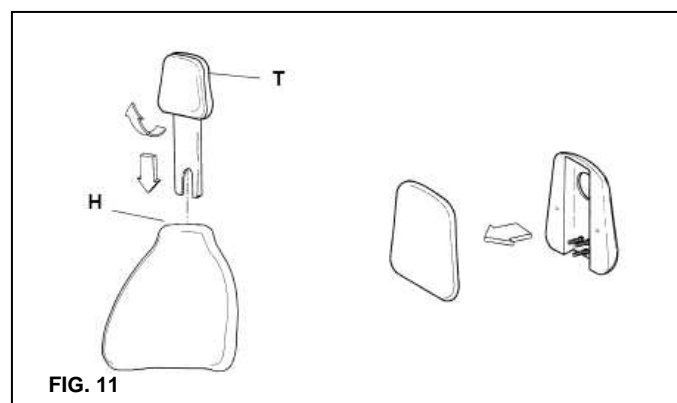
FIG. 11

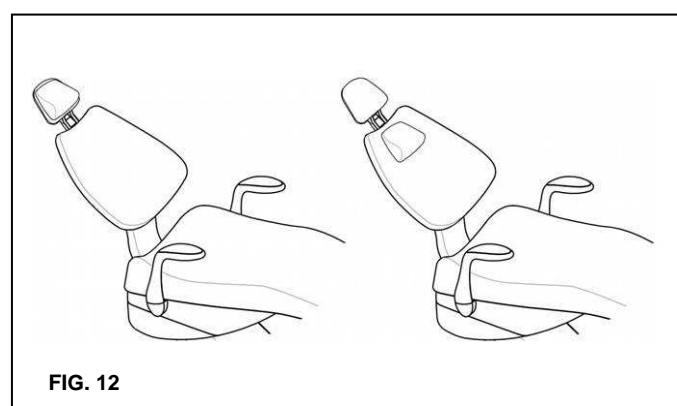
FIG. 12

## 7.2. ARMRESTS.

The Sting chair can be equipped with left or right armrests (see Figure 13).

### OPERATION

To swivel the armrest, pull it upwards (holding it from its bottom part) until the mechanism is unlocked and swivel it outwards, until it reaches the stroke limit. To move the armrest back to its original position, turn it inwards until it gets automatically locked.

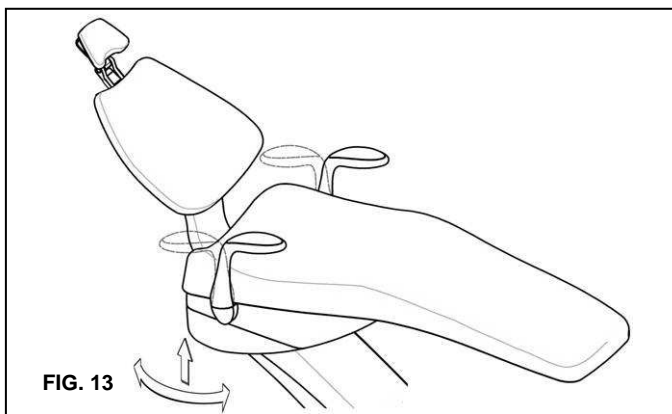
FIG. 13

## 7.3. PROGRAMS.

Upon request, at the moment of the order, the Sting chair can be equipped with a programming device which allows the operator to program three different working positions, which can be activated at any time. For operation, please refer to paragraph 4.3.2.

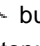
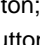
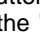
## 7.4. TRENDELENBURG POSITION.

Upon request, the Sting chair can be equipped with a movement kit which allows the chair to reach the Trendelenburg position.

## 7.5. REMOTE CONTROL.

The remote control (Figure 14) is used for moving the chair and programming it.

### OPERATION

- To make the chair go down, keep pressed the  button;
- To make the chair go up, keep pressed the  button;
- To rise the backrest, keep pressed the  button;
- To lean the backrest, keep pressed the  button;
- To let the chair return to "zero" position, tap the "Ø" button;
- To allow the mouthwash, tap "S" button;
- To put the chair in emergency position, press "E" button;
- To activate programming function, press "M" button (for programming, see paragraph 4.3.2);
- To let the chair reach desired programmed working positions, tap "P1", "P2" or "P3" buttons.

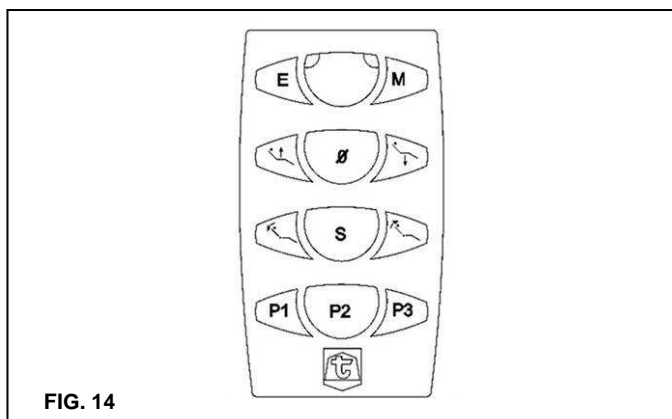
FIG. 14

### BATTERIES REPLACEMENT

1. Remove the hexagonal screw located in the back side of the remote control.
2. Remove the back cover of the remote control.
3. Replace the batteries with new ones, having the same characteristics (2 x AAAA, 1.5V).
4. To re-install the back cover, follow the inverse sequence of the above steps.

### CAUTION.

Remove the battery if the remote control will not be used during a long time period.

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Tecnodent S.r.l.